

decimal Bolus Designer Keyboard & Mouse Controls

The following page details the shortcut keys provided in the user interface of the decimal eRT app and Bolus Designer app. The shortcuts are detailed in their appropriate section of use.

Mouse Controls

Display Views	
Key	Description
Double Left Click	Reset Zoom (fit to scene display)
Right Click + Drag Up	Zoom in
Right Click + Drag Down	Zoom out
Middle Click + Drag	Pan
Left Click + Drag	Rotate (3D only)

Keyboard Controls

General	
Key	Description
[Toggle expansion of left side task menu
]	Toggle expansion of right side task menu
Ctrl + =	Increase font size
Ctrl + +	Increase font size
Ctrl + -	Decrease font size
Ctrl + 0	Reset font size to default
F1	Open the user guide to the appropriate article based on the current block/task
F5	Manual Forced refresh of application and data
F11	Toggle full screen view
3D Views	
Key	Description
Ctrl + 1	Reset the camera to view from the front perspective
Ctrl + 2	Reset the camera to view from the left perspective
Ctrl + 3	Reset the camera to view from the right perspective
Ctrl + 4	Reset the camera to view from the back perspective
Ctrl + 5	Reset the camera to view from the top perspective
Ctrl + 6	Reset the camera to view from the bottom perspective
Ctrl + 7	Reset the camera to view from the isometric perspective

From:

<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

http://apps.dotdecimal.com/doku.php?id=bolusdesigner:userguide:keyboard_controls

Last update: **2022/12/12 12:52**

