

# .decimal Dicom Receiver

A Windows computer must be used to run the AE Title programs. Our current configuration uses a standard Windows 7 Professional PC that automatically logs in with a default user upon startup and the DICOM receiver is included in the computer startup folder. Any setup that ensures the receiver is always running should be sufficient however.

## Setup

The following section describes the dependencies and steps to install and configure the .decimal dicom receiver for Thinknode.

### Python Verion Required:

Python 34

### Packages required to run:

- requests
- colorama
- jsonpickle
- msgpack

```
python -m pip install requests
python -m pip install colorama
python -m pip install jsonpickle
python -m pip install msgpack-python
```

## Installation

Create a shortcut of the receiver\_service.exe and copy the shortcut to the Start Up folder such that it begins on startup.

## Usage

Configure your export client to export to the set IP address and Port as set in the receiver\_config.json file.

From:

<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

[http://apps.dotdecimal.com/doku.php?id=dicom:dicom\\_receiver](http://apps.dotdecimal.com/doku.php?id=dicom:dicom_receiver)

Last update: **2021/07/29 18:21**

