

.decimal Dicom Receiver

Setup

The following section describes the dependencies and steps to install and configure the .decimal dicom receiver for Thinknode.

Python Verion Required:

Python 34

Packages required to run:

- requests
- colorama
- jsonpickle
- msgpack

```
python -m pip install requests
python -m pip install colorama
python -m pip install jsonpickle
python -m pip install msgpack-python
```

Installation

Create a shortcut of the receiver_service.exe and copy the shortcut to the Start Up folder such that it begins on startup.

From:

<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

http://apps.dotdecimal.com/doku.php?id=dicom:dicom_receiver&rev=1460407190

Last update: **2021/07/29 18:19**

