

astroid Dicom App User Guide

Overview

The astroid Dicom App is used to provide access to functions which are used for reading a dicom file and returning more useable standard data types, such as those from the RT Types App available on the thinknode framework. The astroid Dicom App device is not an interactive end user application. Users of the system will write scripts or use a fully interactive software program that makes calls to the functions provided by the astroid Dicom App. The core functionality includes parsing dicom RT_Plan, Structure Set, CT Image, and Dose file types. The Dicom App adheres to the Dicom PS3.3 2015c dicom specification.

Access to the astroid Dicom App is provided by the thinknode™ framework using http json formatted requests. thinknode™ provides the 'backbone' used to send and receive requests, maintain users, realms and organizations, and provide data storage and management.

The [Dicom Manifest Guide](#) contains a complete list of all functions and types supported by the astroid Dicom App through API calls.

Purpose

The purpose of this document is to provide guidance on the setup, access, and usage of the astroid Dicom app.

Getting Started



The primary purpose of the astroid Dosimetry App is to provide a library of efficient and easily accessible proton dosimetry functions which can serve as the foundational tools for developing higher level radiation treatment planning and analysis software programs. The Dosimetry App is not intended to be an interactive end-user application; instead it will serve as a source of tested and validated calculation functions which software developers can utilize in other end-user applications. The astroid Dosimetry App provides access to these calculation functions through the thinknode™ framework which is accessible over the internet and performs all calculations in a cloud based environment. thinknode provides robust, redundant data storage, ensuring industry leading data integrity as well as stable software version management and release process.

Connecting to thinknode™

The Dicom App, and all its released versions live on thinknode™ hosted on the Amazon S3 service. Communication with the App is performed over the internet through thinknode https requests which are

password protected per user and permission based by thinknode organization administrators. The thinknode startup procedure includes full installation of the appropriate application environment onto the virtual servers, which includes hardware and performance checks to ensure sufficient resources are available for the requested application. A full thinknode™ userguide is found at <https://developers.thinknode.com>.

Basic information needed to communicate with the Dicom App through thinknode™:

- thinknode™ user name and password as a base64 encoded blob (e.g.: userName:PassW0rd1 → dXNlck5hbWU6UGFzc1cwcmQx)
- thinknode™ api url (e.g.: <https://api.thinknode.com>)
- thinknode™ version the application is released on (e.g.: v1.0)
- application name (e.g.: Dicom)
- thinknode™ realm name

See [thinknode™ Examples](#) for sample code projects for connecting to the Dicom App.

Precautions

It is the responsibility of those utilizing this application to ensure all that all usages of this product relating to dicom information handling are performed by trained and qualified personnel only and that such personnel is aware that the quality of any generated output data is highly dependent on the quality and correctness of the input data; therefore if any questions or uncertainties exist regarding the quality, units, or identification of input data arise, they must be investigated and resolved before the data are used.

Functions

The Dicom App exposes three primary classes of functions through its API: dose calculation functions (DCF), design task functions (DTF), and radiotherapy support functions (RSF). Each function has its own type of input argument specifications and output results specifications. It is worthy of pointing out the interdependence of these function classes upon one another. Specifically, the DCFs rely heavy on the use of the RSFs and may take as input, the output devices produced by the DTFs. The DTF are typically higher level functions that will modify data parameters and make repeated use of the various DCFs and RSFs to accomplish some type of “optimal” device or plan design.

- Details of each function, argument parameters, and return values are provided at the [Dicom Manifest Guide](#).
- Notes about manifest documentation:
 1. Previous versions of the dicom documentation can be found at the [Dicom Versions](#) page.

Data Types

Details of each datatype used by the Dosimetry App can be found at the [Dicom Manifest Guide](#).

Examples

thinknode™ calculation provider

Official thinknode™ user guide documentation can be found at <https://developers.thinknode.com>. Below are basic examples of communicating with thinknode™ calculation providers and using the Dicom App calculation functions.

- **Python:** Sample Python project (note: this example requires third party libraries) that details the following thinknode™ api functions:
 - [Python: Post to Immutable Storage](#)
 - [Python: Post a Calculation Request](#)

System Usage

Normal system usage requires following the specifications for functions and data types as detailed by the [Dicom App Manifest Guide](#). Making correct calls to the calculation provider requires following the specifications for api requests as detailed by the [thinknode™ API guide](#). Additionally, as most large data structures are returned in base64 encoded format, users can and should perform error checking during decoding to be certain data was not lost or corrupted during transmission.

Details about the intended use of the functions can be found at the [Functions](#) section.

Improper System Usage

When using the astroid Dicom App, as with any complex program, there is the potential for misuse. The thinknode™ framework and Dicom App limit most of the potential misuse of exposed functions and data types, but there's still a potential for misinterpretation of the results provided by the app. Below are a few examples of topics that are easily confused or overlooked when performing even basic tasks.

Known Application Limitations

Below are listed the known application limitations, defects, or inconsistencies.

1. None at this time

Should you discover or suspect any additional errors or limitations not noted above, please contact the .decimal astroid Dicom App development lead, Kevin, Erhart, at kerhart@dotdecimal.com with details of your concerns. All concerns will be addressed as quickly as possible and follow up with the submitter will

occur throughout the inquire and correction process.

USR-006

.decimal LLC, 121 Central Park Place,
Sanford, FL. 32771

From:
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
<http://apps.dotdecimal.com/doku.php?id=dicom:userguide:userguide&rev=1436533426>

Last update: **2021/07/29 18:21**

