

ElectronRT DICOM Receiver

The following guide details how to download, install, and configure the ElectronRT DICOM Receiver to automatically accept and import files into the ElectronRT application for import.

DICOM Receiver

Installation and Setup

- **Download**

1. Download the ElectronRT DICOM Receiver via



- **Installation**

1. Extract the ElectronRT_dicom_receiver.zip file to an appropriate directory of your choosing (e.g.: C:\ElectronRT_DICOM_Receiver)
2. Edit the receiver_config.json file to set the following fields:
 1. **local_ae_title**: The name of the listener (Note: this is not explicitly enforced, it's only for reference)
 2. **local_port**: The local port for the listener to monitor on
 3. **timeout**: The timeout, in seconds, for which to wait for files/connections
 4. **storage_location**: The shared network location to save files to that the ElectronRT

app will be able to access and monitor (see



link)

ElectronRT DICOM Monitoring

From:

<https://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

https://apps.dotdecimal.com/doku.php?id=electronrt:userguide:dicom_receiver&rev=1615928189

Last update: **2021/07/29 18:22**