

decimal eRT Settings

The following settings are present within the decimal eRT settings UI accessed within the View → Settings menu:

App Settings

- **Landing Page**
 - Allows the user to specify whether or not to view the application landing page when decimal eRT is opened.
- **Inactivity Timeout**
 - Specifies the time out period (in minutes) to automatically log the user out of the application. Timeout is based on mouse/keyboard interaction within the user interface. Note: This inactivity timeout is different than the user session token received from the [decimal Launcher](#).
- **Default DICOM Directory**
 - Sets the default DICOM import directory. This directory will be used each time a DICOM import is attempted.
- **UI Theme**
 - Set the visual theme of the application (Note: the 'blue' theme is the currently supported theme. The 'dark' theme is a deprecated theme and not routinely updated.

Local Disk Cache

The local disk cache is where calculation

Network Disk Cache

Patient Storage

From:
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:app_settings&rev=1598983005

Last update: **2021/07/29 18:24**

