

# decimal eRT Settings

The following settings are present within the decimal eRT settings UI accessed within the View → Settings menu:

## App Settings

- **Landing Page**
  - Allows the user to specify whether or not to view the application landing page when decimal eRT is opened.
- **Inactivity Timeout**
  - Specifies the time out period (in minutes) to automatically log the user out of the application. Timeout is based on mouse/keyboard interaction within the user interface. Note: This inactivity timeout is different than the user session token received from the [decimal Launcher](#).
- **Default DICOM Directory**
  - Sets the default DICOM import directory. This directory will be used each time a DICOM import is attempted.
- **UI Theme**
  - Set the visual theme of the application (Note: the 'blue' theme is the currently supported theme. The 'dark' theme is a deprecated theme and not routinely updated.

## Local Disk Cache

The local disk cache is where calculation results are stored on the client computer. The recommended storage size for this cache is 25GB.

Refer to [Data Management Storage and Caching](#) for more information.

## Network Disk Cache

## Patient Storage

From:  
<http://apps.dotdecimal.com/> - decimal App Documentation

Permanent link:  
[http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:app\\_settings&rev=1598983390](http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:app_settings&rev=1598983390)

Last update: 2021/07/29 18:24

