

Patient Courses

Once you have a Patient imported you will need to set up a Course for that patient before being able to create any treatment plans. A Course is used to define the patient's anatomy (e.g.: a single DICOM Structure Set and CT Image Set).

The Course Information settings decided here will be inherited by any plans in this Course and can be edited in the Course UI later to update all non-approved plans within the Course.

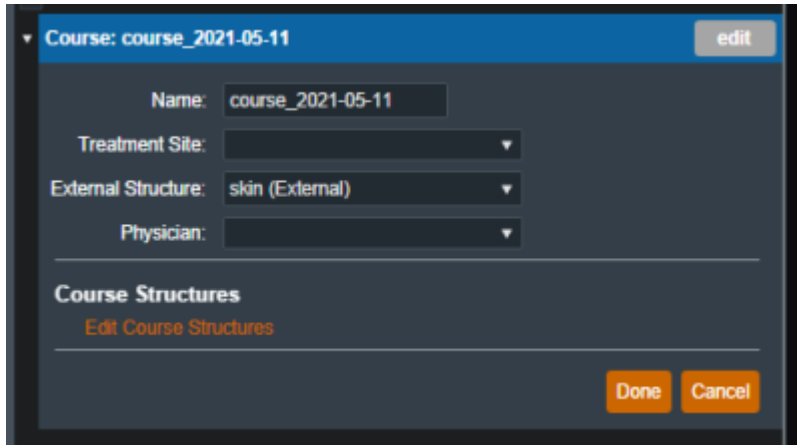
The screenshot shows a software interface for creating a new course. At the top, there is a header bar with a dropdown menu showing 'Course: course_2021-05-11' and an 'edit' button. Below this, the form is divided into several sections. The first section contains four fields: 'Name' with a text input containing 'course_2021-05-11', 'Treatment Site' with a dropdown menu, 'External Structure' with a dropdown menu showing 'skin (External)', and 'Physician' with a dropdown menu. Below these fields is a section titled 'Course Structures' with a link 'Edit Course Structures'. At the bottom right of the form are two orange buttons labeled 'Done' and 'Cancel'.

Fig. 1: Empty Course Block

Course

In the Course block you can set the following fields:

- **Name** : Initially this label is automatically generated but can be changed as desired.
- **Treatment Site** : This list is taken from your site settings, you are able to add or remove treatment sites from the Site Configurations block. For more information please refer to the [Site Settings](#).
- **External Structure**: The external structure will be automatically determined based on DICOM structure types during import. You are also able to override the initial selection with any other structure in the structure list if desired.
- **Physician** : This list is taken from your site settings, you are able to add or remove physicians from the Site Configurations block. For more information please refer to the [Site Settings](#).

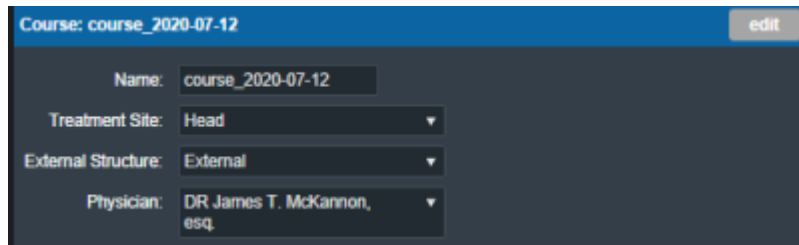
The screenshot shows the same software interface as Fig. 1, but with the course filled out. The header bar now shows 'Course: course_2020-07-12' and an 'edit' button. The 'Name' field contains 'course_2020-07-12'. The 'Treatment Site' dropdown is set to 'Head'. The 'External Structure' dropdown is set to 'External'. The 'Physician' dropdown is set to 'DR James T. McKannon, esq.'. The 'Course Structures' section and the 'Done'/'Cancel' buttons are still present at the bottom.

Fig. 2: Course Example

Course Structures

Certain tasks within eRT require structures to be set to a specific type, and their type is set based on the DICOM RT ROI Interpreted Type (3006,00A4) tag during initial patient import. The Course Structures user interface allows users to change the type of each structure within the patient course.

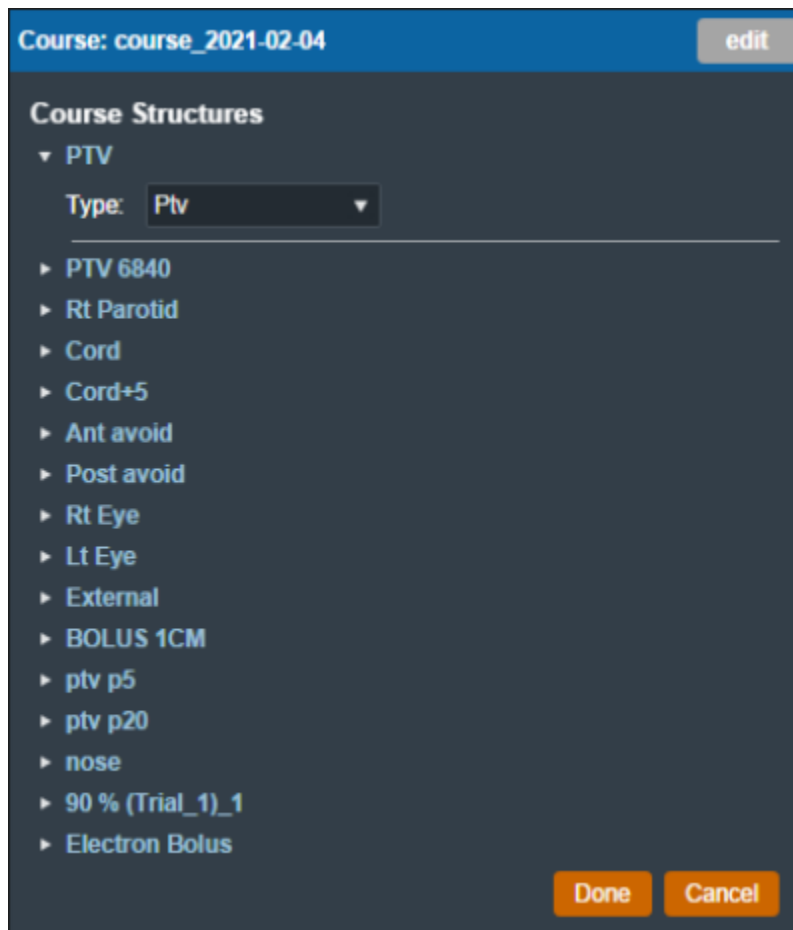


Fig. 3: Editing Course Structures

Note: The RT ROI Interpreted Type (3006,00A4) tag is optional in a DICOM RT Structure Set. As such, the structure type drop down may be empty if no structure type was specified in the imported DICOM files.

From:
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
<http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:courses>

Last update: **2021/07/29 18:28**

