

Patient Courses

Once you have a Patient imported you will need to set up a course for that patient before being able to create any plans. There are two main sections of the course block, the Course and Prescription. The settings decided here will be inherited by any plans in this course and can be edited later.

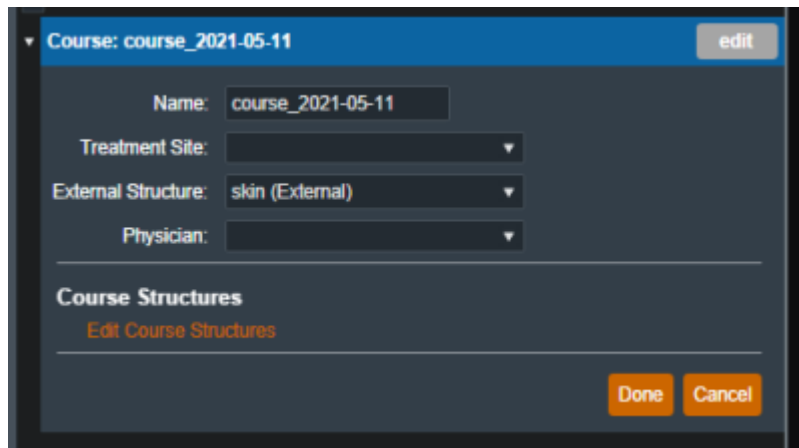
The screenshot shows a web interface for creating a course. At the top, there's a header bar with a dropdown menu showing 'Course: course_2021-05-11' and an 'edit' button. Below this, the form has several fields: 'Name' with a text input containing 'course_2021-05-11', 'Treatment Site' with a dropdown menu, 'External Structure' with a dropdown menu showing 'skin (External)', and 'Physician' with a dropdown menu. Below these fields is a section titled 'Course Structures' with a link 'Edit Course Structures'. At the bottom right, there are two orange buttons: 'Done' and 'Cancel'.

Fig. 1: Empty Course Block

Course

Here in the course block you can set the following fields:

- **Name** : Initially this label is automatically generated but can be changed as desired.
- **Treatment Site** : This list is taken from your site settings, you are able to add or remove treatment sites from the site configurations block. For more info please see <link to site config wiki>
- **External Structure**: If you set an external in the patient import then this will be automatically selected. You are also able to override the initial selection with any other structure in the structure list if desired.
- **Physician** : This list is taken from your site settings, you are able to add or remove physicians from the site configurations block. For more info please see <link to site config wiki>

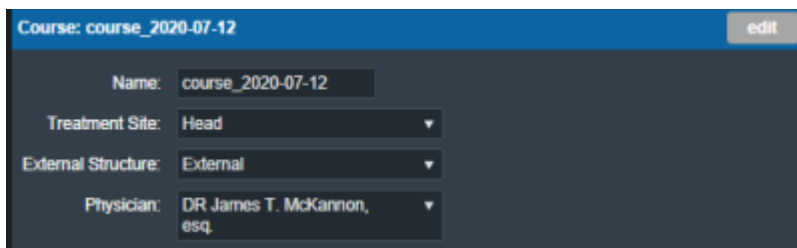
The screenshot shows the same web interface as Fig. 1, but with the form filled out. The 'Name' field contains 'course_2020-07-12'. The 'Treatment Site' dropdown is set to 'Head'. The 'External Structure' dropdown is set to 'External'. The 'Physician' dropdown is set to 'DR James T. McKannon, esq.'. The 'edit' button is still present in the top right corner.

Fig. 2: Course Example

Prescription

You can add multiple prescriptions of different target structures and all of them will be passed into

created plans.

Here in the prescription block you can set the following fields:

- **Label** : A specific name for this prescription.
- **Prescription** : the prescription value (between 0 and 100).
- **Fractions** : How many fractions the prescriptions have.
- **Description** : A description for the prescription.

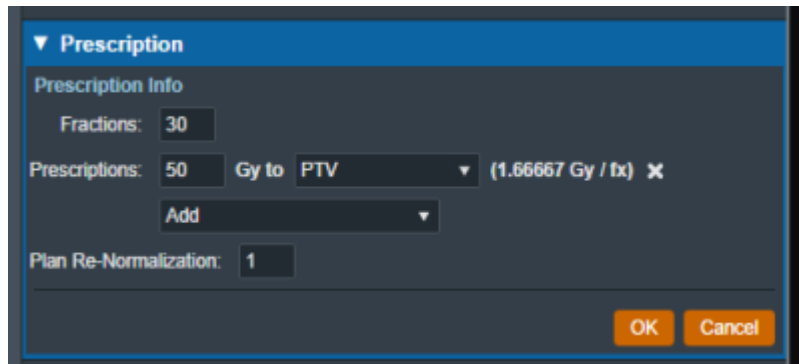


Fig. ##: Prescription Example

From:

<http://apps.dotdecimal.com/> - decimal App Documentation

Permanent link:

<http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:courses&rev=1594667797>

Last update: 2021/07/29 18:24

