

# Patient Courses

Once you have a Patient imported you will need to set up a course for that patient before being able to create any plans. There are two main sections of the course block, the Course and Prescription. The settings decided here will be inherited by any plans in this course and can be edited later.

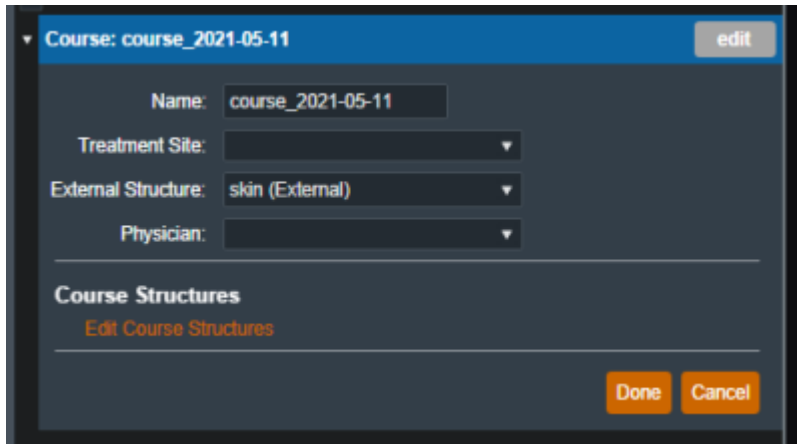
A screenshot of a software interface for creating a new course. At the top, a blue header bar contains the text 'Course: course\_2021-05-11' and an 'edit' button. Below this, the form has several fields: 'Name' with the value 'course\_2021-05-11', 'Treatment Site' with a dropdown arrow, 'External Structure' with the value 'skin (External)' and a dropdown arrow, and 'Physician' with a dropdown arrow. A section titled 'Course Structures' is below these fields, containing a link 'Edit Course Structures'. At the bottom right are 'Done' and 'Cancel' buttons.

Fig. 1: Empty Course Block

## Course

Here in the course block you can set the following fields:

- **Name** : Initially this label is automatically generated but can be changed as desired.
- **Treatment Site** : This list is taken from your site settings, you are able to add or remove treatment sites from the site configurations block. For more info please see <link to site config wiki>
- **External Structure**: If you set an external in the patient import then this will be automatically selected. You are also able to override the initial selection with any other structure in the structure list if desired.
- **Physician** : This list is taken from your site settings, you are able to add or remove physicians from the site configurations block. For more info please see <link to site config wiki>

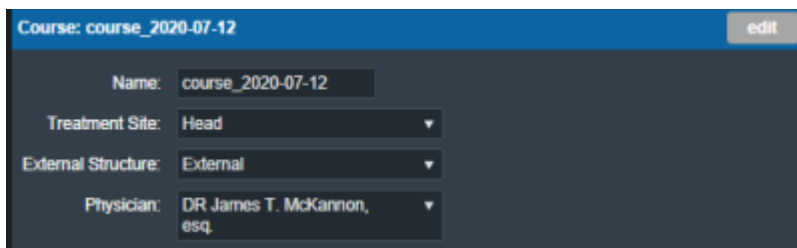
A screenshot of the same software interface, but now filled with data. The header bar shows 'Course: course\_2020-07-12' and an 'edit' button. The 'Name' field contains 'course\_2020-07-12'. The 'Treatment Site' dropdown is set to 'Head'. The 'External Structure' dropdown is set to 'External'. The 'Physician' dropdown is set to 'DR James T. McKannon, esq.'. The 'Course Structures' section and 'Done/Cancel' buttons are also visible.

Fig. 2: Course Example

## Prescription

You can add multiple prescriptions of different target structures and all of them will be passed into

created plans.

Here in the prescription block you can set the following fields:

- **Label** : A specific name for this prescription.
- **Prescription** : the prescription value (between 0 and 100).
- **Fractions** : How many fractions the prescriptions have.
- **Description** : A description for the prescription.

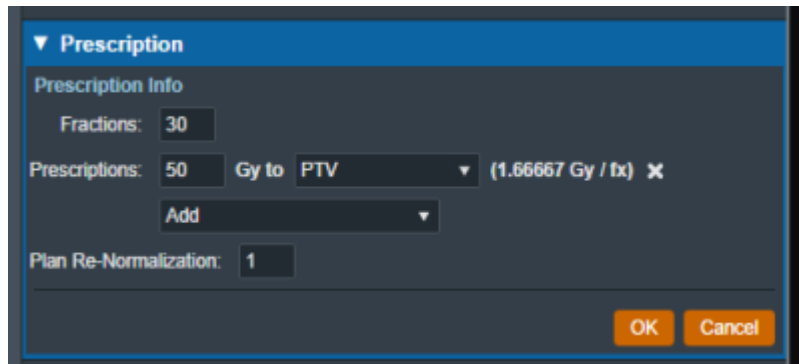


Fig. ##: Prescription Example

From:

<http://apps.dotdecimal.com/> - decimal App Documentation

Permanent link:

<http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:courses&rev=1594667814>

Last update: 2021/07/29 18:24

