

# DICOM Patient Import

## Importing a New Patient



If you have a set of Ct Images and structures you can import them directly into a new patient through the “Import Patients” menu.

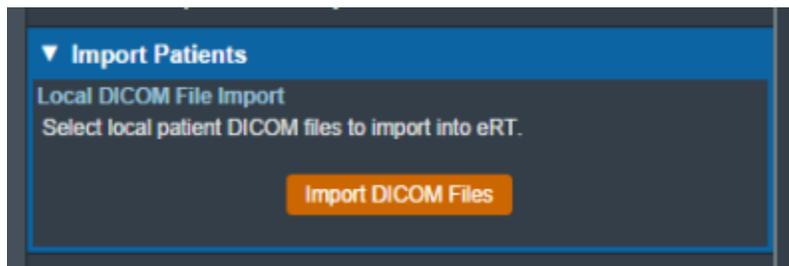


Fig. ##

Selecting the “Import New Patient” option will open the Importing UI.

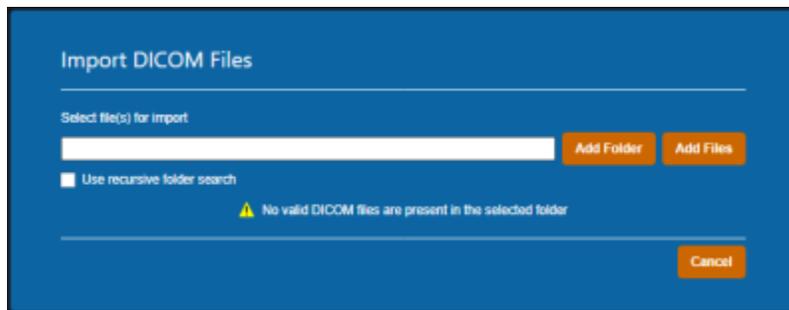


Fig. ##

Here you may either browse your computer or copy the path to the folder that contains the CT images and structures you wish to import.

Then the import will process:

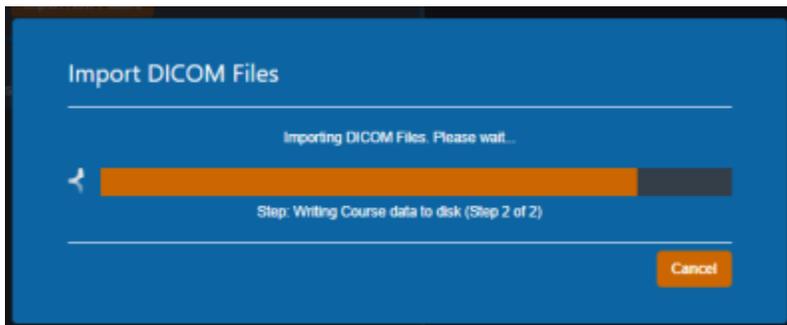


Fig. ##

If there are no Errors while importing you will be taken to a confirmation page for your import.

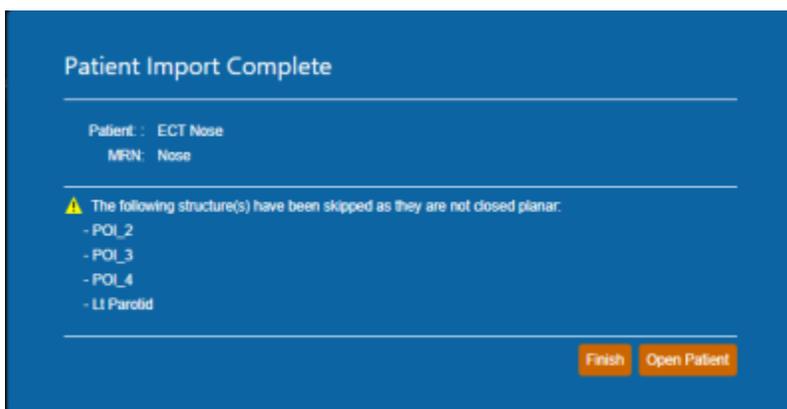


Fig. ##

If there are errors please refer to the [Common Errors](#) section below.

## Common Errors



From:  
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:  
[http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:dicom\\_import&rev=1595436518](http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:dicom_import&rev=1595436518)

Last update: **2021/07/29 18:24**

