DICOM Patient Import

Importing a New Patient



If you have a set of Ct Images and structures you can import them directly into a new patient through the "Import Patients" menu.



Fig.

Selecting the "Import New Patient" option will open the Importing UI.



Fig.

Here you may either browse your computer or copy the path to the folder that contains the CT images and structures you wish to import.

Then the import will proccess:

	Importing DICOM Files, Please wait	
-		
	Step: Writing Course data to disk (Step 2 of 2)	

Fig.

If there are no Errors while importing you will be taken to a confirmation page for your import.

Patient: :	ECT Nose	
MRN:	Nose	
A The follow	ving structure(s) have been skipped as they are not closed planar.	
- POL_2		
- POL 3		
- POL4		
- LI Parolid		

Fig.

If there are errors please refer to the Common Errors section below.

Common Errors



From: https://apps.dotdecimal.com/ - decimal App Documentation

Permanent link: https://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:dicom_import&rev=1595436518

Last update: 2021/07/29 18:24