

Ordering Devices

Once a plan has been approved, the user is able to order the devices that were added to the beams in the plan. Ordered devices will be sent to .decimal for manufacturing through decimal Direct via secure HTTPS.

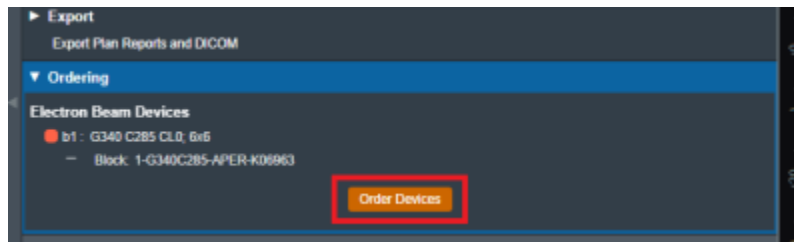


Fig. 1: Part Ordering Example

Select the “order devices” option and all devices will be shown to the user along with the beam in the plan that utilizes said Device. From here users can select which devices they would like to order and move on to the next step.

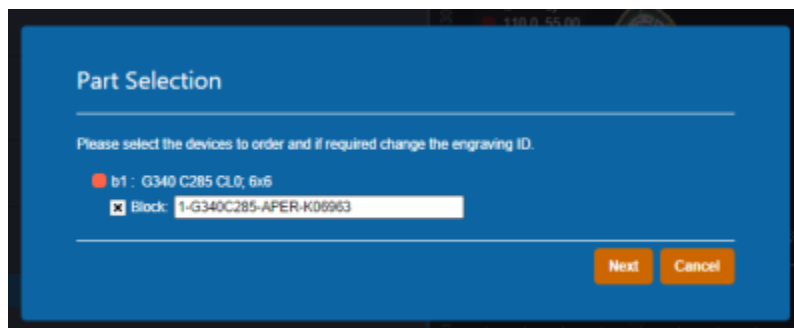


Fig. 2: Part Selection Example

Once the user has decided which devices to order, the following fields will need to be filled out in order to complete the order.

- **Shipping Address:** The address the devices will be shipped to. The areas listed here can be edited or new ones added from the [Organization Configuration](#) block.

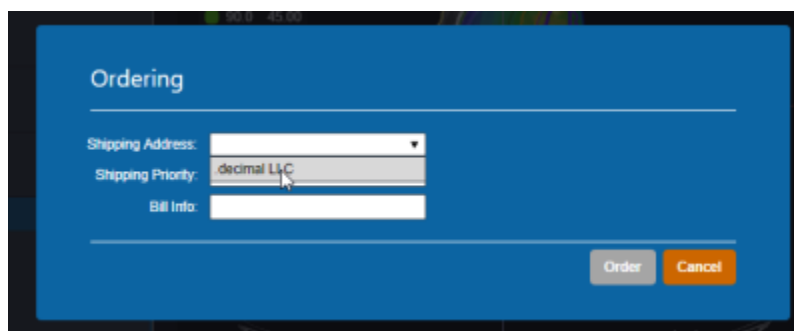


Fig. 3: Ordering Info Example

- **Shipping Priority: Standard** (Overnight delivery by end of next business day) or **Rush** (Overnight delivery before noon of the next business day).
 - **Note:** Extra charges may apply for Rush delivery.

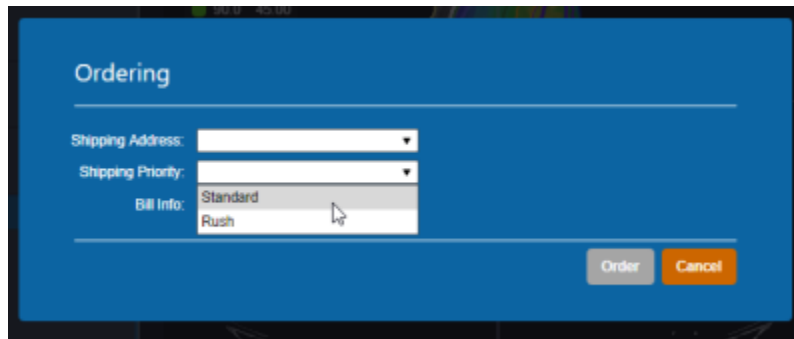
A screenshot of a web application's 'Ordering' form. The form has a blue header with the title 'Ordering'. Below the header, there are three dropdown menus: 'Shipping Address:', 'Shipping Priority:', and 'Bill Info:'. The 'Bill Info:' dropdown is currently open, showing two options: 'Standard' and 'Rush'. At the bottom right of the form, there are two buttons: 'Order' (grey) and 'Cancel' (orange).

Fig. 4: Shipping Priority Selection

- **Bill Info:** Any additional PO or specific Billing information to be added to the order. This is an optional field.

Once these fields have been filled the device(s) can be ordered and the user will be provided with a confirmation if the order is completed successfully.

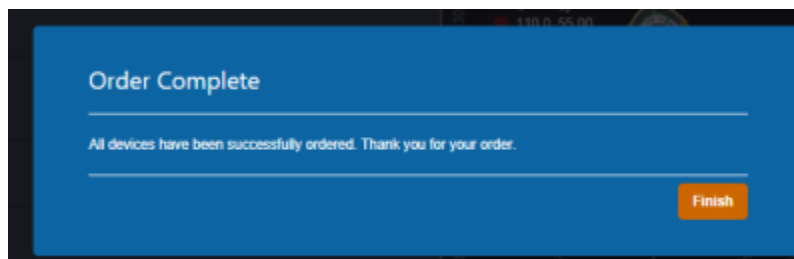
A screenshot of a web application's 'Order Complete' confirmation screen. The screen has a blue header with the title 'Order Complete'. Below the header, there is a message: 'All devices have been successfully ordered. Thank you for your order.' At the bottom right, there is an orange button labeled 'Finish'.

Fig. 5: Order Confirmation

After the order has been placed to [decimal Direct](#) the ordered devices will be noted within the Ordering block. Additionally users can view the order report for that device by selecting the highlighted link. This will take the user to the [decimal Direct](#) page with the order confirmation for that device.


A screenshot of a web application's 'Ordering' block. The block has a blue header with the title 'Ordering'. Below the header, there is a section titled 'Electron Beam Devices'. Under this section, there are two items: 'b1: G340 C285 CL0, 6x6' (marked with a red dot) and 'Block: 1-G340C285-APER-K06963' (marked with a green checkmark). To the right of the 'Block' item, there is a red box containing the text 'Order Report'. At the bottom of the block, there is an orange button labeled 'Order Devices'.

Fig. 6: Access Order Report

From:
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:ordering_devices&rev=1627583286

Last update: **2021/07/29 18:28**

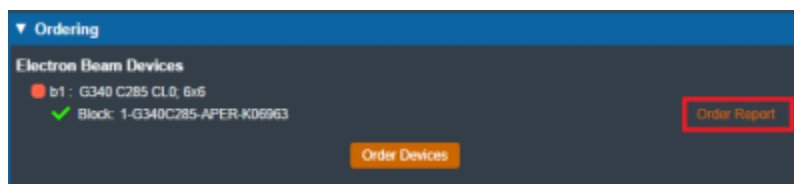