# **Organization Configuration**

The ElectronRT App allows for users with certain permissions to view and edit organization level configuration settings. The Organization Configuration block is located in the main page of the app and is only visible to users with Physics or higher level permissions.

| Patient Search  |                        |                   |    |
|---|------------------------|-------------------|----|
| aucht Scarch  |                        |                   |    |
| Search for patien   | s in the system.       |                   |    |
| Import Patients   |                        |                   |    |
| Select DICOM pa   | tients to import.      |                   |    |
| Organization C  | onfiguration           |                   |    |
| ▼ Organization S  | ettings                |                   |    |
| Organization Level  | Settings               |                   |    |
| Add or edit settings  | or your entire orgar   | ization, includin | g: |
| <ul> <li>CT Override mate</li> <li>Electron Device I</li> </ul> | rials<br>Naterials     |                   |    |
| <ul> <li>Import</li> </ul>                                      |                        |                   |    |
| Export Logs   |                        |                   |    |
| Site Facility Settings  |                        |                   |    |
| Select a Site ID to ed  | t settings for a facil | ty                |    |
| Site ID:  |                        |                   |    |

Fig. 1: Organization Configuration UI

## **Organization Settings**

The Organization Settings block allows the user to view and edit settings that affect the entire organization. These settings include organization name, PDF report logo, CT override materials, and electron device materials.



#### Fig. 2: Organization Settings

While editing the settings in the Organization Configuration block, the UI is changed such that the user cannot click on other sections of the UI until confirming edits with "Done" or canceling edits with "Cancel". Some settings are grouped into tabs, such as the "General" and "Materials" tabs in the Organization Settings.

| Canarat Materiale    | Dasa Colore                  |        |  |
|----------------------|------------------------------|--------|--|
| Ormanization Info    |                              |        |  |
| Name                 | .decimal LLC                 |        |  |
| Report Customization |                              |        |  |
| Report Logo:         |                              | Browse |  |
|                      | recommended size: 220 x 50px |        |  |

Fig. 3: Organization Edit UI

## **Materials**

The Materials definitions allow specifying the device materials and CT override materials. These materials will be usable across the entire organization and all sites.

The units for the materials are as follows:

• density: g/cm3

| General Materials     | Dose Colors |              |                |   |
|-----------------------|-------------|--------------|----------------|---|
| CT Override Materials |             |              |                |   |
| Name                  | Density     | Stopping Pwr | Scattering Pwr | - |
| BlueWax               | 0.92        | 0.92         | 0.92           | × |
| Water                 | 1           | 1            | 1              | × |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |
|                       |             |              |                |   |

## Fig. 4: Materials Configuration

## **Dose Colors**

The Dose Colors definitions allow users to specify the default dose colors and levels when creating new plans. These values will apply to all plans created within the organization across all sites.

| General | Materials          |      |        |   |  |
|---------|--------------------|------|--------|---|--|
| Color   | Dose Colors<br>RGB |      | Dose % |   |  |
| •       | 139, 0, 0          | Edit | 110    | × |  |
|         | 255, 100, 0        | Edit | 105    | × |  |
| •       | 255, 200, 0        | Edit | 100    | × |  |
| •       | 205, 255, 0        | Edit | 95     | × |  |
| •       | 102, 255, 0        | Edit | 90     | × |  |
|         | 0, 255, 0          | Edit | 80     | × |  |
|         | 0, 255, 102        | Edit | 60     | × |  |
| •       | 0, 255, 204        | Edit | 50     | × |  |
|         | 0, 204, 255        | Edit | 30     | × |  |
|         | 0, 100, 255        | Edit | 20     | × |  |
| •       | 0, 0, 255          | Edit | 10     | × |  |
|         |                    |      |        |   |  |

## Fig. 5: Dose Colors Configuration

Note: The dose color options set will only be applied to newly created plans. For existing plans select the 'reset to defaults' link within the Dose Controls right hand side user interface to revert to the site level defaults.

## **Organization Import/Export**

The organization import and export user interface allows users to backup, copy, or give their organization data to other individuals as needed. The following details are worth noting about this feature:

- The organization data will not be readable as plain text.
- If applicable, when importing an organization file the organization data will be upgraded to the latest version of the data model and automatic upgrades added where appropriate. Refer to the release notes for previous versions of the organization since it was last exported for the values of these upgrades.
- The application will need to be restarted after successfully importing a new organization.

## **Export Logs**

The Export Logs block allows the user to export a file containing logs of user activity within the app. This .csv file contains data exported from the app database that keeps track of critical user activity, including (but not limited to): opening of patients and plans, plan approvals, report/DICOM exports, and hardware ordering.



Fig. 6: Export Logs UI

## Site Facility Settings

The Site Facility Settings section of the Organization Configuration allows users to view and edit settings for each site in the organization. After selecting a Site ID, the site level settings are divided into five

categories: Site Settings, DICOM Settings, CT Curves, QA Options, and Machine Settings.



Fig. 7: Site Facility Settings

## Site Settings

The Site Settings block allows for the viewing and editing of miscellaneous site level settings including the site address, physicians, and treatment sites. When editing the site address, the UI has an option to set the address to the one assigned to the site in decimal Direct. In order to edit physicians, the user must first click on the physician name on the list of physicians on left side UI and then click on "Edit Physician" on the right side UI (this is also the case with treatment sites). Users can add or remove physicians and treatment sites using the left side UI.

| anima factorali  | - 0 ×  |
|--|--|
| decinal an   | Alteria lingues Firmes (1966) 1976. 😝 🚺            |
| Comparison Configuration     Organization Configuration     Organization Configuration     Organization     Organization     Organization     Organization     Organization     Organization     Organization     Organization     Organization     Organization | Set lefting  |
|  | Ana tanàna Ang |
| DCOR Million     CT Garon     Corpore     Corpore     Corpore     Municipa Corpore     Municipa Corpore  |  |

Fig. 8: Site Settings UI

## Plan Defaults / Templates

#### **Beam Normalization**

The user can assign a beam normalization template when editing a treatment site. This template will be used to set the normalization during automatic beam calculations for plans in the courses that are assigned to that treatment site.

| Treatment Site Info |                     |          |             |  |  |
|---------------------|---------------------|----------|-------------|--|--|
| Name:               | Head and Neck       |          |             |  |  |
| Plan Defaults       |                     |          |             |  |  |
| Beam Normalization: | Absolute Dose:      | 100 %    | of Fox dose |  |  |
|                     | Normalization Type: | isodose  | •           |  |  |
|                     |                     | Isodose: | 90 %        |  |  |
|                     |                     |          |             |  |  |

## Fig. 9: Treatment Site Settings UI

## **DICOM Settings**

The DICOM Settings block allows the user to view and edit settings related to the export & import of DICOM files. These settings include the default DICOM export directory, a list of DICOM export server AE titles, and the monitoring directory for DICOM Receiver imports.

These settings are applied and available to all users of the selected Site ID for which the settings are present.



Fig. 10: DICOM Settings UI

| DICOM Export Directo | ry   |
|----------------------|--|
| Export Directory     | Sets the default export folder when exporting DICOM files to disk. |
| DICOM Export Server  | Settings   |

| DICOM Export Directo  | ory                              |   |
|-----------------------|----------------------------------|---|
|                       | A list of DICOI<br>ElectronRT ap | M AE titles (DICOM Receivers from another system) that the p can export to.                     |
| Export Export Sorvorc | Server Name:                     | The name of the DICOM server that will displayed to the user when exporting within the eRT app. |
| Export Export Servers | IP Address:                      | The local network IP address to which DICOM files will be sent by the eRT DICOM sender.         |
|                       | Port:                            | The local network port on which the DICOM sender should transmit the DICOM files.               |
| DICOM Import Server   | Settings                         |   |
| DICOM Import          | Sets the moni                    | toring directory for importing patients received from the                                       |

## **CT Curves**

The CT Curves block allows for the viewing and editing of CT conversion curves. These curves must contain data for both relative stopping and scattering power. The relative stopping power data for the selected curve is displayed on a graph on the right side UI. When adding a new CT conversion curve, the user can select and import a CSV file that fully defines the curve values. If there are no curves present, the user is able to obtain a sample CT curve file to have an example of the CSV formatting and values. Additionally, each CT conversion curve in the site can be exported as a CSV file.



## Fig. 11: CT Curves UI

The CT Curves are defined as json as shown in the below example (note: this data is provided as a user guide reference only and should not be used in a commissioned treatment planning system; as such, it does not contain full values of CT curve data):

## CT Curve Example

```
{
    "scattering_power_curve": [
        {
```

```
"key": -1000.0,
      "value": 0.0010
  },
   {
      "key": -706.0,
      "value": 0.2920
   },
],
"stopping_power_curve": [
   {
      "key": -10000.0,
      "value": 0.0010
   },
   {
      "key": -1000.0,
      "value": 0.0010
   },
]
```

## **CT Image Window and Level Presets**

The CT Curves block allows for the viewing and editing of CT image window and level value presets. These presets will appear as options in the right side CT Controls for electron plans in the selected site.

| Name:   | Bone |  |  |  |
|---------|------|--|--|--|
| Mindow: | 2500 |  |  |  |
| Level   | 500  |  |  |  |



## **QA** Options

}

The QA Options block allows the user to view and edit settings related to Dose QA. The UI allows the

import of a CT image set that defines a custom water phantom. The user must define a surface entry point and select a CT curve when importing a custom phantom. Settings used for the default water phantom include the dimensions of the phantom, dose grid spacing, and the default measurement plane depth.

| 🖬 noine factorill   |  |            | - 0 X                                 |
|---|--|------------|---------------------------------------|
| Danitur View  |  |            |                                       |
| Jectrol an  |  |            | Albenh bingen Kirara (1960) 12368 😛 📳 |
| Palent South     Teel to get the system   | GA Options<br>Castaling GA Plantes<br>Castal Descript Of Images Impose<br>Castal Descript Of Images Impose | -          |                                       |
| Import Patients     Send DCCMultimer & equit.   | Caller Bass Qi-CT Care<br>School: Plater Planton   | GE 10      |                                       |
| Organization Companion     Organization Settings  | Detact Measurement Plans Ought<br>Webs Plantari Starts   | 100<br>100 |                                       |
| <ul> <li>Experiinge</li> <li>Sour Jones Te des proposes (socie)</li> <li>Sour Jones Te des proposes (socie)</li> <li>Sources Te des proposes (socie)</li> </ul> | Vide Planet Ilin V<br>Vide Planet Ilin J   | Be<br>Be   | -                                     |
| e Sac Seringa<br>In DCCM Seringa  |  |            |                                       |
| • CT Carea  |  |            |                                       |
| On Options     Edit for GA Options for the summer site  |  |            |                                       |
| + Martin Inline   |  |            |                                       |
|   |  |            |                                       |
|   |  |            |                                       |

Fig. 13: QA Options UI

## **Machine Settings**

The Machine Settings block allows the user to view, add, clone, and edit machines for the selected site. The machine settings are divided into five tabs: General, Geometry, Applicators, Commissioning, and Advanced.

## **General Machine Settings**

The General tab of the Machine Settings block allows for the viewing and/or editing of general machine information such as the machine name, description, serial, type, and physical SAD.

| 🖬 desired Redevel   |   | - п к                            |
|---|---|----------------------------------|
| Jugan Yan   |   |                                  |
| Jechai ell  |   | ithea flague fitnes SMD 1010 😅 🚺 |
| <ul> <li>Fallent Saarsk<br/>Saarsk is palarisk in the system</li> </ul>         | Devel Decello Aphales Contenents Alexand<br>Review Name |                                  |
| + Import Palantis<br>Salart MilliNpalants to import                             | Socyales Ascreptor<br>Social Astronomics                |                                  |
| * Organization Configuration  | Photod INC: 100 all                                     |                                  |
| <ul> <li>Dynamics beings</li> </ul>   | Last Modes By: Albeits Hagen Revea                      |                                  |
| Expertispe  |   | •••                              |
| Unit Facility Uniterative<br>National Vision Circle and Control prior a Landary |   | _                                |
| No: 10/0004 +   |   |                                  |
| + Six Sellegs   |   |                                  |
| <ul> <li>DOM Service</li> </ul>   |   |                                  |
| + CTCarren  |   |                                  |
| + GA Epilera  |   |                                  |
| * Michael Satage  |   |                                  |
| No. two Selfrey<br>Ind a real Nation  |   |                                  |
|   |   |                                  |
|   |   |                                  |

Fig. 14: General Machine Settings UI

#### 10/12

#### Machine Geometry Settings

The Geometry tab of the Machine Settings block allows the user to view and edit Machine (Equipment) coordinate system settings. These settings include the reference gantry angle, reference couch angle, reference collimator angle, and the rotation direction of each axis, relative to IEC 61217 coordinate system.

| and device and the second of the second seco |   | - D K                            |
|--|---|----------------------------------|
| Septem Vigo  |   |                                  |
| .dectrical an  |   | Allanda Yalayara Pilanaa 🛛 🖬 🔁 📑 |
| <ul> <li>Failent Sarch<br/>Sarch le palerts inthe spilen.</li> </ul>   | Denses Densem Associate Communities Advected              |                                  |
| <ul> <li>Import Palanda<br/>Autori Ini (Kr) palanta in report</li> </ul>   | Surgi visulari<br>Esci Rufunge Beg<br>Esci Rufunge Caster |                                  |
| V Organization Configuration   | CARGO INTAGE ING  |                                  |
| + Dyanianise Belleys   | Dallmate Robert Dealthe Oxfordia                          |                                  |
| Expertisge   |   | ····                             |
| New Yorking Services.<br>Reserve the Gran and Antioperior a landing  |   |                                  |
| Siz: 100cadeal +   |   |                                  |
| + Die belings  |   |                                  |
| <ul> <li>DOM Settings</li> </ul>   |   |                                  |
| + ETGamm   |   |                                  |
| + GA Epitora   |   |                                  |
| * Machine Sattings   |   |                                  |
| Northern Sulfrage<br>Tell a real Naciones  |   |                                  |
|  |   |                                  |
|  |   |                                  |
|  |   |                                  |



#### **Applicator Settings**

The Applicators tab of the Machine Settings blocks allows for the viewing and/or editing of applicator and beamline device settings. Information regarding blocks, intensity modulators, and applicators of varying sizes are displayed and the user can choose which applicators are available when creating a treatment plan using the current machine. The available applicators are dependent on type of machine (Siemens, Varian, or Elekta) chosen during the creation of a new machine.

While users are able to change the default target margin of the block, it should be noted that users are unable to change manufacturer specific settings (such as physical block size) as these have been preconfigured and validated by .decimal. 11/12



Fig. 16: Applicator Settings UI

#### **Commissioning Data Settings**

The Commissioning tab of the Machine Settings block allows the user to view and edit commissioning data by energy. The user can view the nominal energy and R90 values of the current commissioning data or import new commissioning data from a local file. The ElectronRT App currently only has support for commissioning data import from the Pinnacle treatment planning system.

The Commissioning tab also allows the user to import MU dose conversion data tables and display them in the UI. Each MU data file must contain the Output Factor and Air Gap Factor value tables for a given beam energy. The user is able to obtain a sample MU conversion table file to have an example of the CSV formatting and values.

| animal Restort  |  | - D ×                                 |
|---|--|---------------------------------------|
| Senior New  |  |                                       |
| Jectral an  |  | Alfreds Magner Flows (2002) 12340 🖨 📘 |
|   |  |                                       |
| <ul> <li>Paleri Garch</li> <li>José la sérait la la seraite</li> </ul>        | General Generals Agabation Communities Advanced  | °                                     |
|   | Seried (seg. 6 Mil   |                                       |
| <ul> <li>Import Patients</li> <li>Salect Int INUMERIA Report.</li> </ul>      | Service Lines  |                                       |
| * Ergenineten Configuration   | NET DECKA  |                                       |
| <ul> <li>Dependention Ratilings</li> </ul>                                    | Hit 404th  |                                       |
| • Expectiope  | His Line   |                                       |
| Size Faculting Cardingen<br>Securit a Size Of to call cardings for a Sacility | Bill SSTun   |                                       |
| No: 1000adand +   | E Herrinel Ellerger 1 Mel  |                                       |
| * Do beings   | r Honinal Energy 1 MaY<br>Collected Door, 4 201 SeA40                                  |                                       |
| <ul> <li>DOM Settings</li> </ul>  | CADA FACE TARE   |                                       |
|   | Age (Ref Table)  |                                       |
| P LI Came   |  |                                       |
| <ul> <li>GA Epitora</li> </ul>  | 20 CR1 CR1 CR1 CR1 CR1   |                                       |
| * Machine Sellings  | 1949 1981 1981 1982 1923 1934  |                                       |
| The last lattice is   | 10-10 13011301109811001100   |                                       |
| int and failure   | 2620 1001100 100 100 100   |                                       |
|   | 2021 100 100 100 100   |                                       |
|   | A0 Dig 1909 1999   |                                       |
|   | THAT EACH AND THE THE THE THE THE  |                                       |
|   | 30 100 000 100 100 100   |                                       |
|   | 648 1.001 E.001 E.003 E.003  |                                       |
|   | 16-10 1.001 E.HU E.HE E.HU E.HU  |                                       |
|   |  |                                       |
|   | 2020 100 100 100 100   |                                       |
|   |  |                                       |
|   |  |                                       |
|   | <ul> <li>Remain Lineage of Barry</li> <li>Encoded and the State State State</li> </ul> |                                       |
|   | Terral Date:   |                                       |
|   |  | _                                     |
|   |  |                                       |



## Advanced Settings

The Advanced tab of the Machine Settings block allows for the view and editing of advanced machine

configuration parameters. These settings include tolerance tables, wedge tray information, blocking tray information, and room imaging parameters. These values are typically not used within the treatment planning process, but are included in DICOM Plan export to meet end user DICOM RT Plan requirements.



#### Fig. 18: Advanced Machine Settings UI

