

Plan Approval

Before users are able to order blocks or boluses in the ElectronRT app the plans for those parts but be approved by a user with physician permissions. This action ensures parts are not ordered for plans that have not been properly QA'ed.

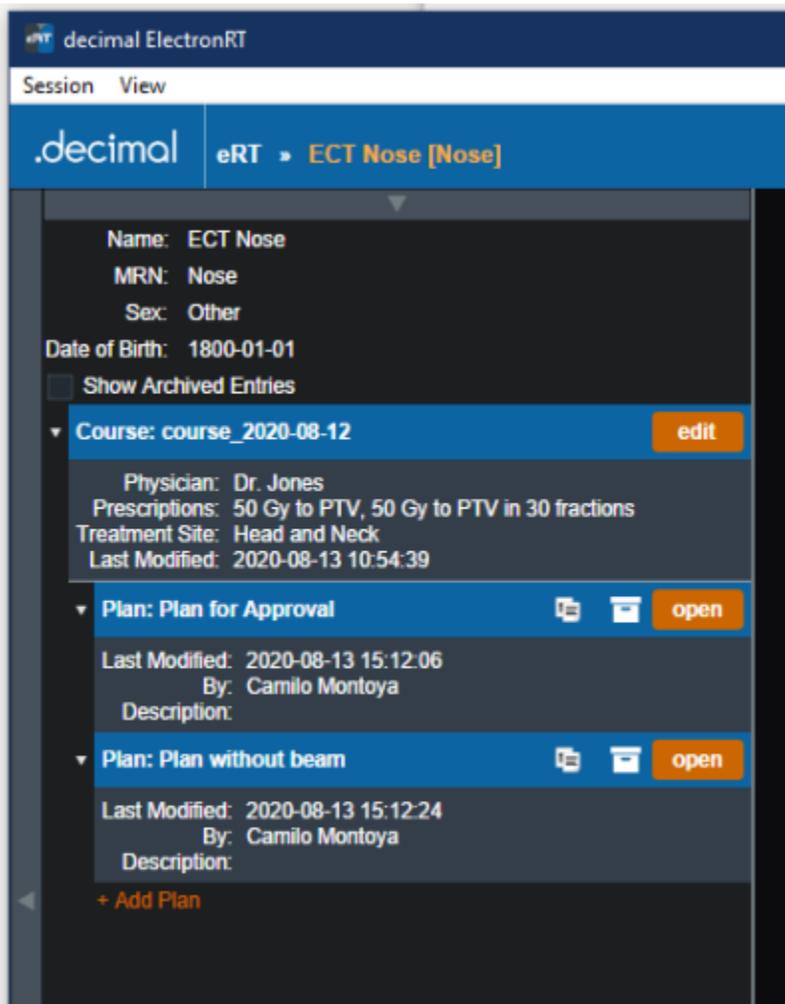


Fig. 1: Import Directory Input

When a user goes to select a plan for a course for their patient they will be able to tell if the plan is approved or not plans that are green are approved while not approved plans are the standard blue.

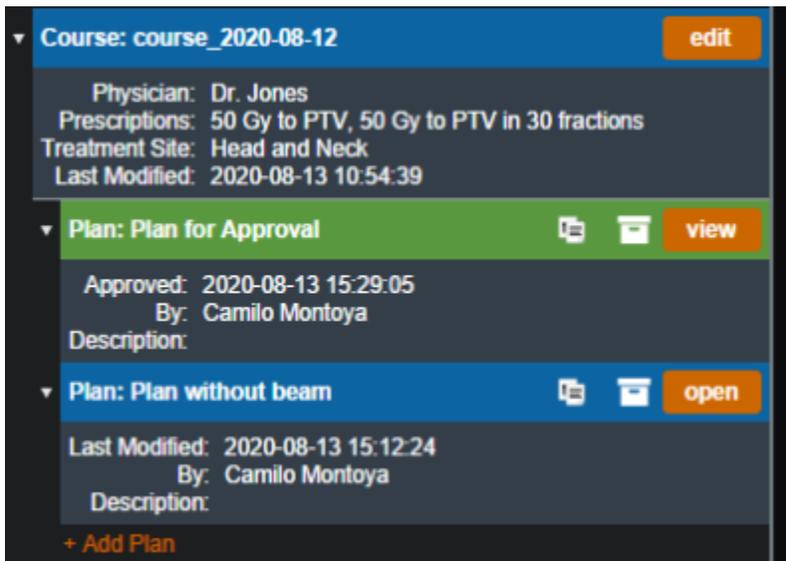


Fig. 2: Import Directory Input

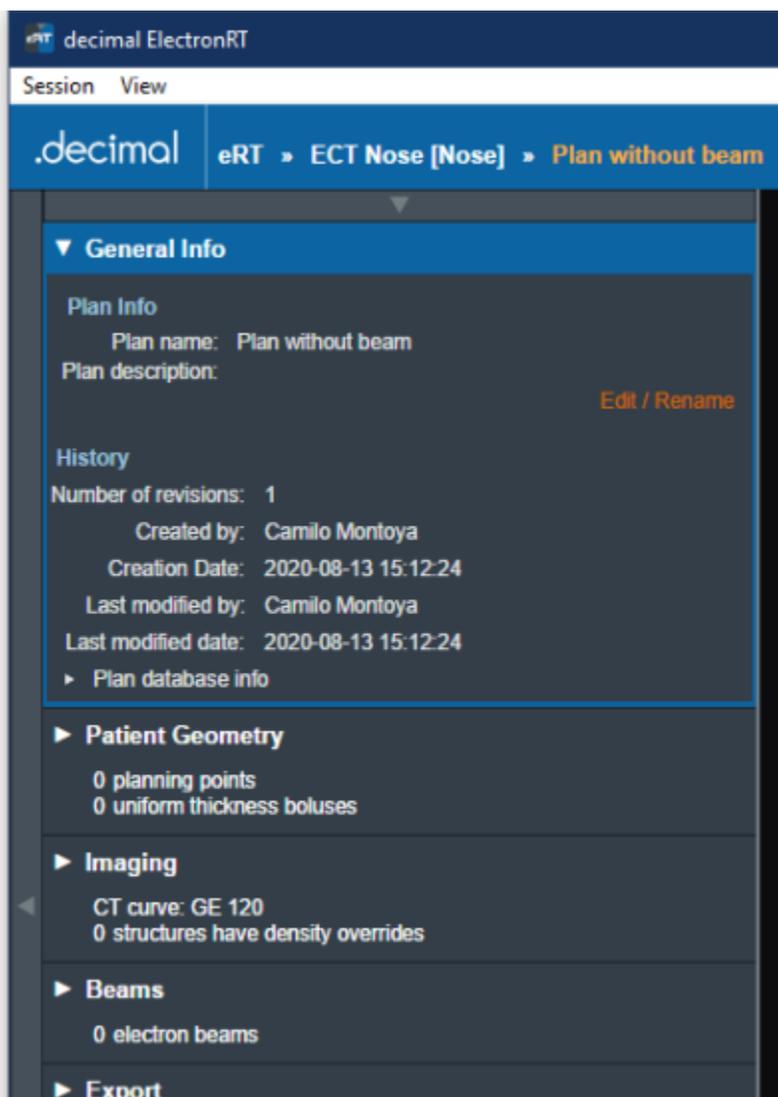


Fig. 3: Import Directory Input

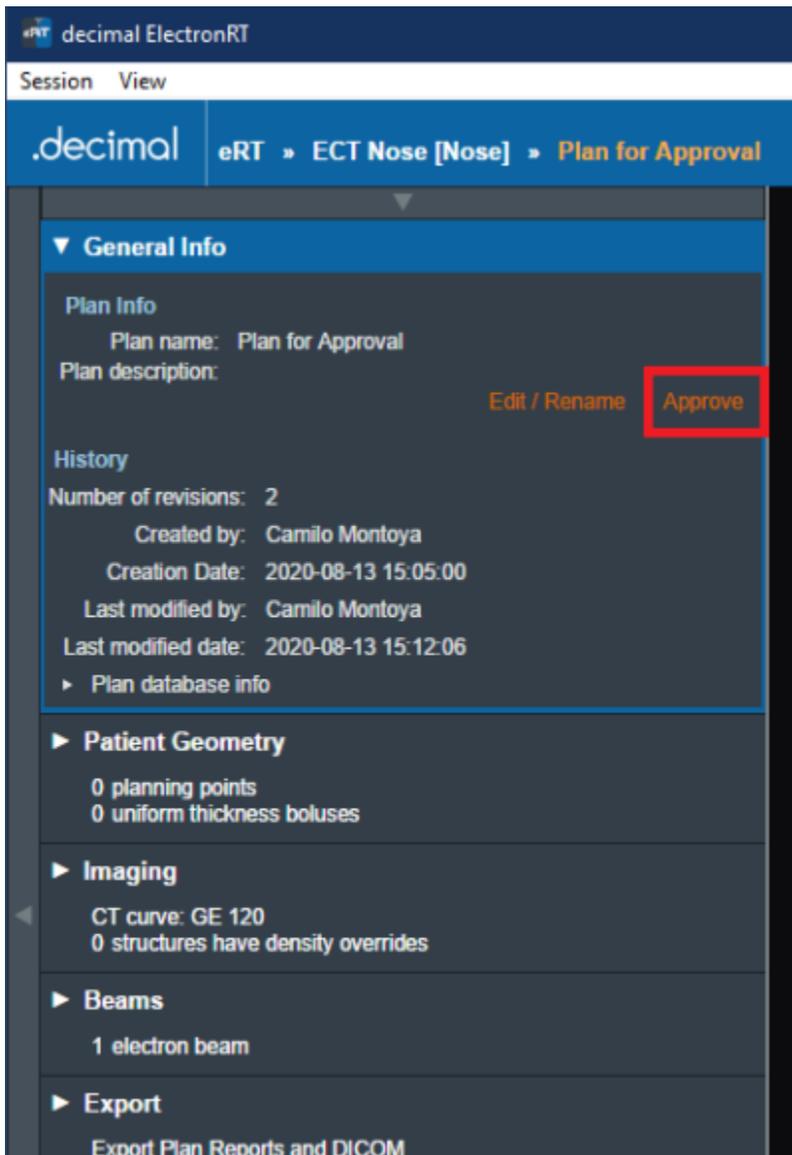


Fig. 4: Import Directory Input


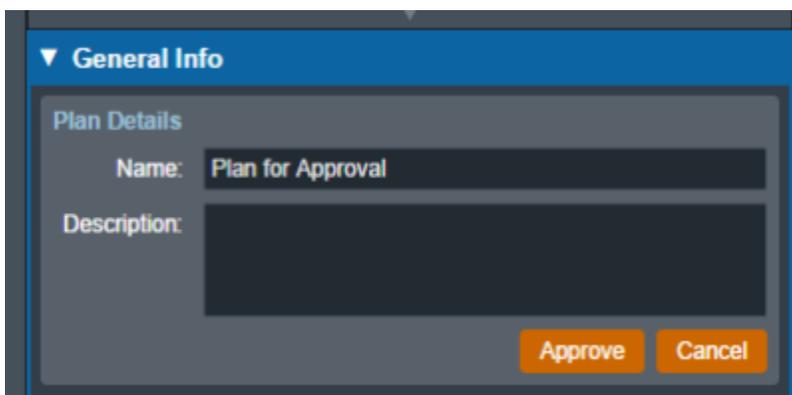


Fig. 5: Import Directory Input

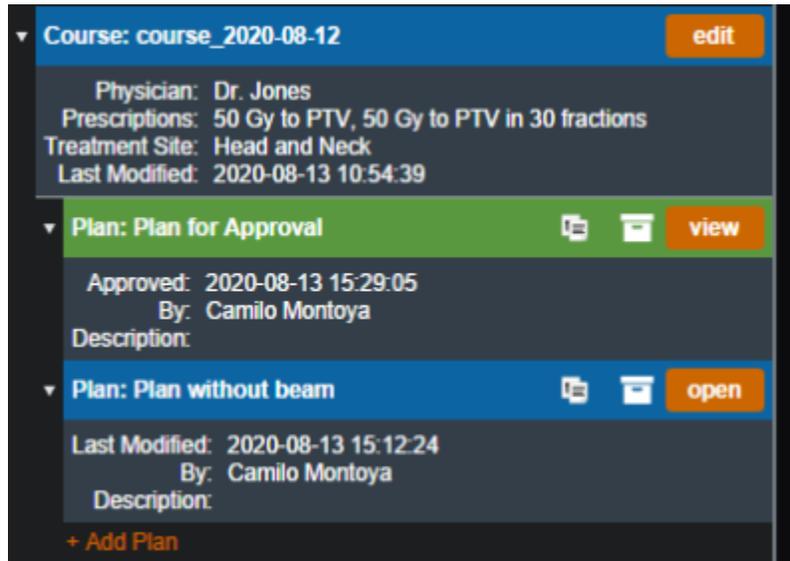

▼ Course: course_2020-08-12 edit

Physician: Dr. Jones
Prescriptions: 50 Gy to PTV, 50 Gy to PTV in 30 fractions
Treatment Site: Head and Neck
Last Modified: 2020-08-13 10:54:39

▼ Plan: Plan for Approval view

Approved: 2020-08-13 15:29:05
By: Camilo Montoya
Description:

▼ Plan: Plan without beam open

Last Modified: 2020-08-13 15:12:24
By: Camilo Montoya
Description:

[+ Add Plan](#)

Fig. 5: Import Directory Input

From:
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:plan_approval&rev=1597352084

Last update: **2021/07/29 18:24**

