The ElectronRT App includes a set of QA Tools that allow the user to measure each treatment beam dose on a water phantom, compute dose on bolus simulation CT images and compare results, and export QA PDF reports. The QA Tools block is only enabled for approved treatment plans.

Dose QA

The Dose QA block computes and displays the dose for each beam in the treatment plan to a water phantom. For each beam, the user is able to define the SSD and the depth of a measurement plane. This measurement plane will be used to take a cross-sectional sample of the water phantom dose. The dimensions of the water phantom, as well as the default measurement plane depth and dose grid spacing, are defined and editable in the Site QA Options. After defining the measurement plane depth for at least one beam, the user will be able to click the Export QA Dose Files button to export a QA Dose DICOM file for every measurement plane dose to a local directory.



Fig. 1: Dose QA

Bolus QA

The Bolus QA block allows the user to import DICOM CT images and an optional structure set used for bolus simulation QA. The process is very similar to the DICOM Patient Import.

1/5

2/5

Re-Import Structures elect file(s) for import				
C:\Users\anegron\Desktop\imetveir	mport/Nose_plan for export_1	Browse		
	10504 004 4474774 4 day			
21-2 16 840 1 114460 178 1 19899 2T-2 16 840 1 114460 178 1 15899	48581.964.1471771.1.00m			
CT-2 16 840 1 114460 178 1 15895	48581 964 1471771 100 dcm			
CT-2 16 840 1 114460 178 1 15895	48581 964 1471771 101 dcm			
CT-2.16.840.1.114460.178.1.15895	48581.964.1471771.102.dcm			
CT-2.16.840.1.114460.178.1.15895	48581.964.1471771.103.dcm			
CT-2.16.840.1.114460.178.1.15895	48581.964.1471771.104.dcm			
CT-2.16.840.1.114460.178.1.15895	48581.964.1471771.105.dcm			
CT-2.16.840.1.114460.178.1.15895	48581.964.1471771.106.dcm			
CT-2.16.840.1.114460.178.1.15895	48581.964.1471771.107.0cm			
31-2 16 840 1 114460 178 1 15895 37 3 46 840 1 114460 178 1 15895	48581.964,14/1//1,108.0Cm			
51-2.10.840.1.114400.178.1.13893 PT-2.16.940.1.114460.178.1.15895	48581.904.1471771.109.00m			
51-2. 10.040.1. 114400.170.1. 10850	40504 054 4474774 440 down			
			Selected file(s):	12
	Import Files			

Fig. 3: Bolus QA Import

Dose is computed on the re-imported CT images using the same beams as the treatment plan. The resulting dose image, as well as the original plan dose and a gamma comparison image, are then displayed in the UI. The user can edit the Bolus QA block controls to change the gamma parameters as well as shift the isocenter for each beam in the re-computed dose to account for positional shifts in the bolus simulation CT images.

The comparison view shows the three images simultaneously on the corresponding CT images as well as a graph displaying the DVHs for the original and computed dose images. The user can switch between Transverse, Sagittal, and Coronal for the comparison view in the Bolus QA Display Options in the right side controls.



Fig. 4: Bolus QA

Bolus QA Reports

The Bolus QA Reports block allows the user to export PDF reports with data gathered from the Bolus QA block. Just like with Plan Reports, the user can add screenshots of the Bolus QA display UI to the Bolus QA report.

3/5

▼ QA Tools									
► Dose QA									
► Bolus QA									
▼ Bolus QA Reports									
Report Screen Shots Add or remove screenshots for exported reports									
TransverseCompare 🗙									
Screen Shot Name:									
Take Report Screen Shot									
Export Bolus QA Report Export the Bolus QA Report (including saved screen shots) Open report after export Copy PDF after export									
Notes: Example notes for Bolus QA report.									
Export Plan QA Report									
► Block QA Reports									

Fig. 5: Bolus QA Reports UI

Bolus QA reports contain basic patient and treatment plan information as well as gamma analysis results, dose reference point data, and structure dose statistics.

decimo	•BT						
Vecinia							
decimal Ele	ctronRT						
Version 0.0.1-2020.16	1						
Bolus QA Rep	ort						
Patient							
Name: IC .decimal ID: C	T Nose HEN 5505 Patient ID	Note Ba	n Date: 1800-1 10 Other	lan-01			
Plan							
Plan Name: Plan Bescription:	Example Plan 1	Plan ID: 1					
Gamma Analy	sis						
P/F Reg. Passis	Rate (%) Actual	Passing Rate	%) Germe Tel	erance i	Nex German		
Fell 95.00	89.83 calles for Balas Cit.		2%/0.20		1.60		
Point Dose Ve	fication						
10	Orbital Date (- Committee	Done (The)				
p1 : Center of PT/ p2 : -4.8, -24.5, 3	47.84	48.39 2.57					
Structure Com	puted Dose M	etrics					
50	His (Sy) Has (Sy	Mean (Sy)	Volume (ccc)				
akan (External)	0.00 82.56	7.39	218.15				
PTV 6840	0.80 70.37	49.95	205.99				
#9v #20	0.44 64.73	38.39	394.02				
ptv p5	0.56 89.49	47.65	151.75				
90 % [THM[_1]_1	0.80 82.55	39.87	29.01				
POLIS ICH	0.00 45.21	1.76	144.10				
Cant	0.40 37.78	5.24	16.93				
Ci040+5	0.26 44.59	7.83	47.16				
Dectron Balas	0.00 59.43	9.38	455.44				
Lt Eye	0.80 27.90	2.01	0.15				
At Des	0.72 26.45	2.42	2.69				
RJ Panelid	0.37 4.18	1.70	13.46				
8088	0.04 69.30	42.55	5.19				
	Employ Tennes		_		giral Territoria	No Tele-	
						+ 80%	
	1 minutes				ine lie		
	1001 0.0				Contraction of the local division of the loc		

Fig. 6: Bolus QA Report

Block QA Reports

The Block QA Reports block allows the user to export PDF printouts of the electron blocks used in the treatment plan. The user can choose to display each block as a physical (true size) or isocenter projection. There is also a printer-friendly option that removes the color from the block projection.



Fig. 7: Block QA Reports UI

Plan QA



Fig. 8: Block QA Report

From: http://apps.dotdecimal.com/ - decimal App Documentation Permanent link: http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:plan_qa&rev=1599069598 Last update: 2021/07/29 18:24