

# Plan Reports

The ElectronRT App allows PDF export of treatment plan reports. Treatment plan reports include all patient, plan, and beam data and can optionally include screen shots taken in the application.

## Report Customization

The ElectronRT App provides options for customizing the exported plan report including: customizing the header logo and setting the different options for details that will appear on the report.

## Header Logo

By default the header logo is set to the logo of the application. The header logo can be optionally overridden by the user in the Organization Configuration block. For more information please see <link to

site config wiki>



«Organization Name»

Patient: Anon A89376  
Plan: Plan 1

 Default App Logo

**decimal ElectronRT**  
Version: 0.0.3

**Treatment Plan Report**

**Patient**  
Name: Anon A89376 HIN: A89376 Birth Date: 1983-Jan-01  
Journal ID: 800003 Patient ID: 1 Sex: Other

**Course**  
Course Name: course\_2020-07-20 Prescription: 54 Gy to PTV in 30 fractions  
Course ID: 1 Treatment Site: Head and Neck  
Physician: Referring Physician

**DICOM Info**  
Structure Set Name: PDIANDR01 Structure Set Label: PLAN\_15  
Patient Position: RFS

**Plan**  
Plan Name: Plan 1 Plan ID: 1  
Plan Description:

**Imaging**  
CT Curve: GE 120

**Beams**  
Machine Model: VarianTrueBeam (Last Modified By: Alfredo Negrin-Rivera On: 2020-06-04 13:37:21)  
Coordinate System: Equipment

**Beam List**

ID	Target	Energy	Gantry	Collimator	Isocenter (cm)	Entrance (cm)	SSD	Aperture
01	G555 CD CLC; 5x6	1800	9 MV	100 deg	0 deg	0 deg	-8.35, -34.45, 7.30	-8.35, -34.45, 7.30 180 cm 8x8

Treatment Approved: Unapproved - Last Modified: 2020-07-20 10:45:35 by Daniel Patenaude  
Report Created: Plan 1\_report.pdf 2020-07-20 11:33:28 by Daniel Patenaude

1/1

From:  
<http://apps.dotdecimal.com/> - decimal App Documentation

Permanent link:  
[http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:plan\\_reports&rev=1595258668](http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:plan_reports&rev=1595258668)

Last update: 2021/07/29 18:24

