Plan Reports

The ElectronRT App allows PDF export of treatment plan reports. Treatment plan reports include all patient, plan, and beam data and can optionally include screen shots taken in the application.

▼ Export
▼ Plan Reports
Report Screen Shots Add or remove screenshots for exported reports
Screen Shot Name:
Take Report Screen Shot
 Export Plan Report Export the Plan Report (including saved screen shots) Open report after export Copy PDF after export Report Content Options Patient Geometry Export Dose Reference Points Imaging Export Density Overrides Beams Export Beam List Use IEC Coordinate System Export Ordered Hardware IDs
Export Plan Report



Report Customization

The ElectronRT App provides options for customizing the exported plan report including: customizing the header logo and setting the different options for details that will appear on the report.

Header Logo

By default, the header logo is set to the logo of the application. The header logo can be optionally overridden by the user in the Organization Configuration block. For more information please see <link to

site config wiki>



The header will be placed as shown in figure 2. The header image is not scaled and is placed on the report at its native size.

ani Plan 1	376										
docim		-									
oecimo	ם וב	· · ·	Default App	Logo							
decimal E	lectro	DNRT									
Version 0.0.3											
Treatment	Plan R	eport									
Patient											
And the second s	Anon A89 826963	376	alient ID.	A89376 1	Birth B Sec.	ate: 150 00	13-Jan-81 Wer				
Course											
Course Name Course 10: Physician:	1 Referin	1020-07	-20 Press Treat	riplies nert Site	14 Gy Head	end Neck	30 Facliens				
DICOM Info											
Structure Set Patient Positi	Name Po	SEANDRO 75	1 Structu	re Set Lat	H PLA	US					
Plan											
Plan Name: Plan Descripti	Plan	1 Pla	n 191 1								
Imaging											
CTOwner G	£ 129										
Beams											
Hachine Hode Coordinate Sy	A va atom: Dr	urlan Trus Julip men	ribeam (Last t	Pedfiel	By: Alfred	o Negran-I	Rivera On: 2038-86-24	13(37)21)			
Beam List											
10 01 : 6356 CO	CL0; 6x6	nese	9 MeV	Santry 356 deg	Creech 0 deg	Cellinato 0 deg	r Taocenter (cm) -0.35, -34.45, 7.50	Extrance (cm) -0.35, -34.45, 7.50	550 188 cm	Applicator Ext	

Fig. 2: Custom Header

Recommended Size

Since the custom header image is not scaled when placed into the report, it's recommended to use a size that does not interfere with the report contents or reduce the readability of the report.

As such, the recommended image size for a custom header logo is:

- Width: 200 to 500 pixels
- Height: 40 to 75 pixels

Larger sized images may still render correctly on the report as long as they fit a standard portrait letter size, but are not recommended since they may impact readability of the report. Images larger than the PDF page may have the image cropped to fit on a single page.

Screen Shots

Users are able to take screen shots to include on their treatment plan report. Screen shots will include the entire display UI including all CT, Dose, 3D, BEV, and statistics views. Once a screen shot has been

captured it will appear on the exported treatment plan report

▼ Plan Repo	orts						
Report Screen Add or remove	Shots screenshots for exported reports	5					
	Ortho+DVH	×					
	DVH+Stats	×					
Screen Shot Name: 3D							
	Take Report Screen Shot						

Fig. 3: Report Screenshots

After specifying a name for the screenshot, the application will allow the user to capture the current display and save the screen shot for the plan report.

Managing Screen Shots

Once a screenshot has been captured the following actions can be performed for each screen shot:

- **Preview/View:** You can view the captured screen shot by selecting the screenshot in the list of saved captures
- **Delete:** You can remove a captured screenshot so that it is no longer exported to the treatment plan report by selecting the 'X' icon next to the screenshot name.

Report Content

By default plan reports will contain all plan information currently present in the treatment plan. Users can optionally chose to not export specific content to the treatment plan report including: dose reference points, density overrides, beams, or ordered hardware information.

Coordinate System (Equipment & IEC)

By default the treatment plan report will export beam geometry information in the equipment coordinate system. Users can optionally chose to export beam coordinates using the IEC coordinate system as specified in the Machine Configuration. This option is provided as a checkbox in the Report Content Options section.

The selected coordinate system option will be displayed on the treatment plan report as either "Equipment" or "IEC" as shown in the figure 4 below.

Blass.									
Plan Name: Plan Description:	Plan 1 Pla	a 30: 1							
Imaging									
CT Carves 08 120									
Bearra									
Machine Model Coordinate System	VarianTea Equipment	Beern (La	at Modified	By: Alfr	do Negron-Ri	vera Or: 2028-06-24	13:37:21)		
Bears List									
10 11 0356 C0 CL0;	Target Luti nose	8 NeV	Gantry 356 deg	Couch 0 deg	Collimator D deg	Teocenter (cm) -0.35, -34.45, 7.50	Entrance (cm) -0.33, -34.45, 7.50	290 100 ore	Applicator
	time .		- 1-			10	-		
						state _	No. of Concession, Name		

Fig. 4: Report Coordinate System

For more information on setting the IEC coordinate systems please see <link to site config wiki>



From: http://apps.dotdecimal.com/ - decimal App Documentation Permanent link: http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:plan_reports&rev=159534916 Last update: 2021/07/29 18:24



Plan Reports