## **Electron Treatment Plans**

Once the patient has a course with a valid prescription, a plan can be created. When adding a new plan to a patient, the user will be prompted to add a name and an optional description for the plan.

Fig. 1: Creating a Plan	

From the Patient Overview, the user can choose to clone an existing plan. As the cloned plan will be a copy of the original plan (except for the plan name and if cloning an approved plan, the clone will be unapproved), the user should name the cloned plan appropriately to avoid confusion.







Fig. 2: Cloning a Plan

	reatment Site: Last Modified: 2	2020-08-24 14:28:00	Clon	e the e	existing t	reatment plan.
•	Plan: Example	e Plan 1			open	
	Last Modified: By: Description:	2020-08-24 14:32:33 Alfredo Negron-Rivera An example plan used for guide purpose	5.			
	+ Add Plan					





Fig. 4: Viewing Inactive Plans

## **Approved Plans**

Refer to Plan Approval for information on approved plans.

## **Plan History**

The plan history UI exists within the *General Info* block of the plan overview. This interface allows users to view the details of all changes to their treatment plan, view the plan at each point and time of each revision (as read-only), and make any past revision active as the current revision.

Refer to figure 5 below for a detailed explanation for each of the Plan History features.

Test Plan 2						
ename Approve	Revision 3: activ label: set modified: 2022 by: Alfre hide { op pa val s }	ve revision 2-04-28 14:01:11 edo Negron-Rivera e details 1 p: update, ath: [ name ], al: { some: Test Plan 2				
	Revision 2: mak label: set modified: 2022 by: Alfre show	ke active 2   view 2-04-28 14:01:00 edo Negron-Rivera w details				
	Revision 1: mak label; set modified: 2022 by: Alfre	view 3 2-04-28 14:00:47 edo Negron-Rivera				



Plan History UI Explained

- [1] **Show Details**: The details of the changes of each revision. Currently the revision comparison shows a textual comparison of the selected revision and the immediate preceding revision.
  - For example: the details shown for Revision #3 tell us that the *Update* operation was performed on the variable *name* to the value *Test Plan2*. Or in other words, the plan's name was updated to 'Test Plan2'.
- [2] Make Active: Makes the desired selected revision the current active revision of this plan.
- [3] View: Opens the selected revision for review by the user in a read-only viewer

From: http://apps.dotdecimal.com/ - **decimal App Documentation** Permanent link:

http://apps.dotdecimal.com/doku.php?id=electronrt:userguide:tutorials:plans&rev=1617067268



Last update: 2021/07/29 18:24