

# p.d Version History



This section is intended to provide access to previous versions of the p.d documentation for releases of p.d after 5.4.0.

---

## Release Notes

### p.d 5.4.2

November 10, 2021

#### *What's New*

- Added button to About window to activate Windows 7 Compatibility Mode.
- Added new diagnostic logging for when p.d successfully authenticates with .decimal remote servers.
- Updated p.d menu icon to make it more clear the button opens a menu.

#### *Bug Fixes*

- Updated p.d to set CT image HU value maximum to 8000 to prevent error from metal artifacts

---

### p.d 5.4.3

January 3, 2022

#### *What's New*

#### *Bug Fixes*

- Fixed a crash when opening the Digitizer Wizard.

## p.d 5.4.1

September 20, 2021

### *What's New*

- Updated all design wizard machine drop downs to only include machines for the current site facility (selected from the decimal Launcher)
- Updated p.d to no longer include or uses addresses from the treatment machine
- Updated p.d ordering address to use the decimal Direct shipping address and disabled changing of the ordering address in the Order Wizard

### *Bug Fixes*

- none
- 

## p.d 5.4.0

August 26, 2021

### *What's New*

- Updated p.d to be installed and launched from the decimal Launcher application
  - Removed the login screen as p.d must always be launched using the decimal Launcher
  - Removed internal update mechanics from p.d as install and updates will be managed by the decimal Launcher
- Updated the Import Wizard machine drop downs to only include machines with the current application site ID (sent from the decimal Launcher)
- Added a new Planning Menu option to take the user to the user profile management page of decimal Direct

### *Bug Fixes*

- none
- 

From:  
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:  
[http://apps.dotdecimal.com/doku.php?id=pdotd:pdotd\\_versions&rev=1641240580](http://apps.dotdecimal.com/doku.php?id=pdotd:pdotd_versions&rev=1641240580)

Last update: **2022/01/03 20:09**

