

p.d Version History



This section is intended to provide access to previous versions of the p.d documentation for releases of p.d after 5.4.0.

Release Notes

p.d 5.4.4

March 23, 2022

What's New

- Removes patient initials from order files and default device engravings
 - A p.d generated anonymized ID is now assigned so that no PHI is transmitted to .decimal servers
- Updated Structure as Bolus to prevent the user from selecting the DICOM external structure as the bolus structure
- Added a “Track Recent Orders” button that takes the user to the Direct Status page
- Added support for international shipping addresses in order files

Bug Fixes

- Fixed a bug that causes p.d to crash when importing some structure sets

p.d 5.4.3

January 3, 2022

Bug Fixes

- Fixed a crash when opening the Digitizer Wizard.

p.d 5.4.2

November 10, 2021

What's New

- Added button to About window to activate Windows 7 Compatibility Mode.
- Added new diagnostic logging for when p.d successfully authenticates with .decimal remote servers.
- Updated p.d menu icon to make it more clear the button opens a menu.

Bug Fixes

- Updated p.d to set CT image HU value maximum to 8000 to prevent error from metal artifacts
-

p.d 5.4.1

September 20, 2021

What's New

- Updated all design wizard machine drop downs to only include machines for the current site facility (selected from the decimal Launcher)
- Updated p.d to no longer include or uses addresses from the treatment machine
- Updated p.d ordering address to use the decimal Direct shipping address and disabled changing of the ordering address in the Order Wizard

Bug Fixes

- none
-

p.d 5.4.0

August 26, 2021

What's New

- Updated p.d to be installed and launched from the decimal Launcher application
 - Removed the login screen as p.d must always be launched using the decimal Launcher
 - Removed internal update mechanics from p.d as install and updates will be managed by the decimal Launcher
- Updated the Import Wizard machine drop downs to only include machines with the current

application site ID (sent from the decimal Launcher)

- Added a new Planning Menu option to take the user to the user profile management page of decimal Direct

Bug Fixes

- none

From:

<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

http://apps.dotdecimal.com/doku.php?id=pdotd:pdotd_versions&rev=1648153017

Last update: **2022/03/24 20:16**

