2025/10/21 03:06 1/3 p.d Version History

# p.d Version History



This section is intended to provide access to previous versions of the p.d documentation for releases of p.d after 5.4.0.

# **Release Notes**

# p.d 5.4.4

March 24, 2022

#### What's New

- Removes patient initials from order files and default device engravings
  - A p.d generated anonymized ID is now assigned so that no PHI is transmitted to .decimal servers
- Updated Structure as Bolus to prevent the user from selecting the DICOM external structure as the bolus structure
- Added a "Track Recent Orders" button that takes the user to the Direct Status page
- Added support for international shipping addresses in order files

#### Bug Fixes

• Fixed a bug that causes p.d to crash when importing some structure sets

# p.d 5.4.3

January 3, 2022

#### **Bug Fixes**

• Fixed a crash when opening the Digitizer Wizard.

2025/10/21 03:06 2/3 p.d Version History

### p.d 5.4.2

November 10, 2021

#### What's New

- Added button to About window to activate Windows 7 Compatibility Mode.
- Added new diagnostic logging for when p.d successfully authenticates with .decimal remote servers.
- Updated p.d menu icon to make it more clear the button opens a menu.

#### Bug Fixes

• Updated p.d to set CT image HU value maximum to 8000 to prevent error from metal artifacts

### p.d 5.4.1

September 20, 2021

#### What's New

- Updated all design wizard machine drop downs to only include machines for the current site facility (selected from the decimal Launcher)
- Updated p.d to no longer include or uses addresses from the treatment machine
- Updated p.d ordering address to use the decimal Direct shipping address and disabled changing of the ordering address in the Order Wizard

#### **Bug Fixes**

none

## p.d 5.4.0

August 26, 2021

#### What's New

- Updated p.d to be installed and launched from the decimal Launcher application
  - Removed the login screen as p.d must always be launched using the decimal Launcher
  - Removed internal update mechanics from p.d as install and updates will be managed by the decimal Launcher
- Updated the Import Wizard machine drop downs to only include machines with the current

application site ID (sent from the decimal Launcher)

• Added a new Planning Menu option to take the user to the user profile management page of decimal Direct

### **Bug Fixes**

none

#### From:

http://apps.dotdecimal.com/ - decimal App Documentation

Permanent link:

http://apps.dotdecimal.com/doku.php?id=pdotd:pdotd\_versions&rev=1648153623

Last update: 2022/03/24 20:27

