

# Astroid Planning Release Notes

**Version:** 2.1.3

**Date:** April 24, 2018

The release version build of the Astroid Planning App version 2.1.3 is now available (note: this version is still pending FDA 510k clearance). No data types or api function signatures are changed in this release. This release includes the following changes since the 2.1.2 release:

- Updates the dependent Dosimetry App version to be 2.1.1-1 from 2.1.0. The Dosimetry App update includes removing the function call `find_potential_max_voxels()` during feasibility and MCO initialization that was causing max constraint violations and significant progress stalling (usually seen at 0.5%).
- Provides better DRR smoothing when a specific drr grid is provided

From:

<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

[http://apps.dotdecimal.com/doku.php?id=planning:release\\_notes:2.1.3](http://apps.dotdecimal.com/doku.php?id=planning:release_notes:2.1.3)

Last update: **2021/07/29 18:23**

