

# Astroid Planning Release Notes

**Version:** 2.3.1

**Date:** Jan 29, 2019

The release version build of the Astroid Planning App version 2.3.1 is now available. No existing ResultsAPI function signatures are changed in this release. This release includes the following changes since the previous release:

- Added exporting of CT images from the export block.
- The BEV view was incorporated into the ResultsAPI viewer.
- Local planning calc performance was reviewed.
  - Background calculation performance improved by lowering priority of background jobs.
  - Improved local calcs when beams are created or added to fraction group by forcing pbs layer creation to be a remote calculation.
- Adds a secondary dose DVH legend overlay.
- The structure visibility toggle has been updated to better show items that are turned off.
- Planning device right hand side controls have been updated to be able to toggle devices independently of beams.

From:

<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

[http://apps.dotdecimal.com/doku.php?id=planning:release\\_notes:2.3.1&rev=1572620129](http://apps.dotdecimal.com/doku.php?id=planning:release_notes:2.3.1&rev=1572620129)

Last update: **2021/07/29 18:22**

