

# Astroid Planning Release Notes

**Version:** 3.0.1

**Date:** Oct 31, 2019

The release version build of the Astroid Planning App version 3.0.1 is now available. No existing ResultsAPI function signatures are changed in this release. This release includes the following changes since the previous release:

- Updates snout and device positioning to properly use the full facility device mount settings
  - No changes were made to the data model, but the “aperture\_mount\_info” and “shifter\_mount\_info” are now properly used and the “downstream\_device\_gap” value has been deprecated
- Adds RC image to the ResultsAPI Viewer display
- Fixes behavior of “Show Structures” checkbox in ResultsAPI Viewer
- Fixes export issue with missing dll files
- Fixes issue with DICOM export option missing for published PBS plans

From:

<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

[http://apps.dotdecimal.com/doku.php?id=planning:release\\_notes:3.0.1](http://apps.dotdecimal.com/doku.php?id=planning:release_notes:3.0.1)

Last update: **2021/07/29 18:23**

