
2025/11/02 07:05 1/1 Astroid Errors

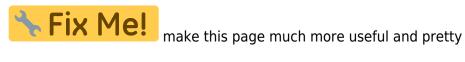

Thinknode Errors

- "provider terminated unexpectedly":
 - During feasibility: calculation grid is too dense and crashed the provider. Make calc grid coarser
- "401":
 - Session token has expired, close planning and relogin.
- "50X"
 - Any 500-504 error codes mean there was a temporary issue on the Thinknode side. Closing, reopening, and retrying what gave the 50X error should resolve the issue.

From:

http://apps.dotdecimal.com/ - decimal App Documentation

Permanent link:

http://apps.dotdecimal.com/doku.php?id=planning:userguide:errors&rev=147076516

Last update: 2021/07/29 18:22

