


# Creating a Plan

1. The point has now been reached where a *Plan* can be created
2. Click on the blue *Add Plan* link under the *Patient Model* to create a new plan
  1. In the box that opens the user should name the plan and add any description they may want

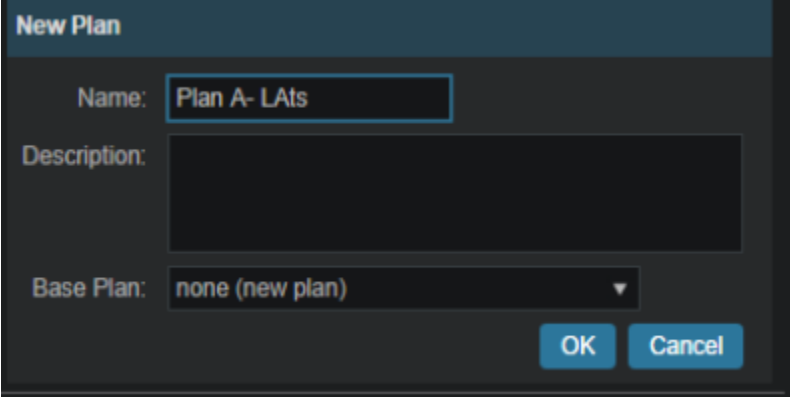

The screenshot shows a 'New Plan' dialog box with the following fields and options:

- Name:** Plan A- LATs
- Description:** (empty text area)
- Base Plan:** none (new plan) (dropdown menu)
- Buttons:** OK, Cancel

2. Note that the *Base Plan* option is used to specify whether an empty plan should be created or if the new plan should be pre-filled using the selected *Plan Template* (details on *Plan Templates* can be found [here](#))
3. Click on the blue *OK* button when finished and the *Plan* has been created
3. Open the plan and begin the planning process by clicking on the blue *Open* button next to the new plan
  1. Note that users are free to have as many plans as desired within a the *Patient Model* and each Plan will specify which portion of the Course Prescriptions it is attempting to implement

From:  
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:  
[http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:creating\\_a\\_plan](http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:creating_a_plan)

Last update: **2021/07/29 18:28**

