


# Creating a New Plan

Once a Planning patient has been added, a new request and plan can be created.

1. From the *Patient list*, select the patient to open.
2. Open the patient by double clicking the patient row or selecting the *Open* button in the *preview UI*.
3. From the *patient overview* select the *Create Request* button.
  1. The *treatment request* and *schedule* can be set from within the *request*.
4. Now that a *request* has been created, select the *Create Plan* button.
  1. Name the plan and add any relevant description.
5. Once the plan is created, select the *open* button next to the plan name.

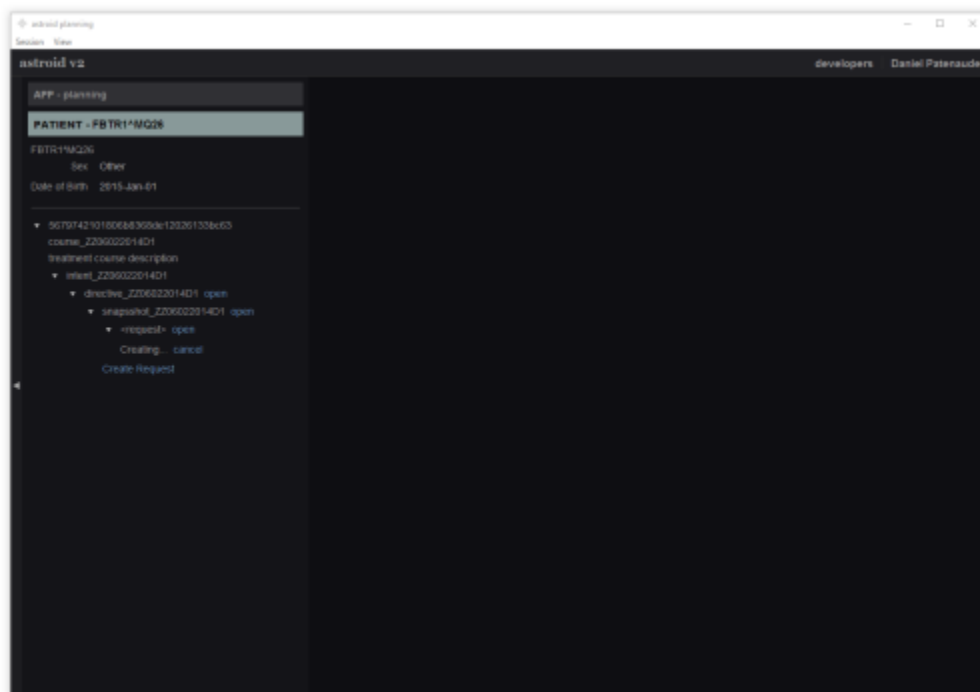

From:  
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:  
[http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:creating\\_a\\_plan&rev=1471437379](http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:creating_a_plan&rev=1471437379)

Last update: **2021/07/29 18:24**

