
Creating a Plan

Once a Planning patient has been added, a new request and plan can be created.

- 1. From the *Patient list*, select the patient to open.
- 2. Open the patient by double clicking the patient row or selecting the *Open* button in the *preview UI*.
- 3. From the *patient overview* select the *Create Request* button.
 - 1. The treatment request and schedule can be set from within the request.
- 4. Now that a request has been created, select the Create Plan button.
 - 1. Name the plan and add any relevant description.
- 5. Once the plan is created, select the *open* button next to the plan name.

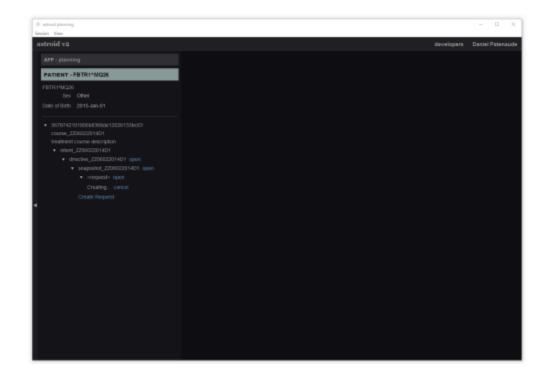

From

http://apps.dotdecimal.com/ - decimal App Documentation

Permanent link:

http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:creating_a_plan&rev=1474387574

Last update: 2021/07/29 18:24

