



Creating a Plan




1. The point has now been reached where a *Plan* can be created - new pics
2. Click on the blue *Add Plan* link under the *Patient Model* to create a new plan

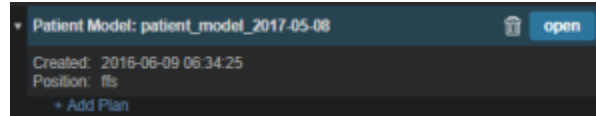

1. In the box that opens the user should name the plan and add any description they may want

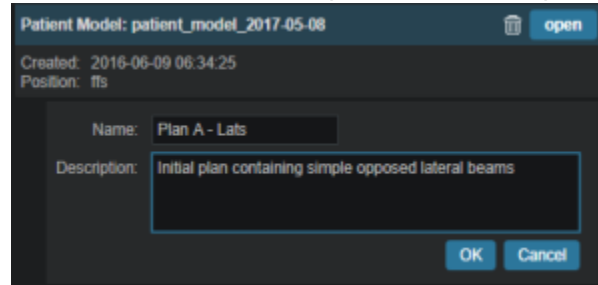

2. Click on the blue *OK* button when finished and the *Plan* has been created
3. Open the plan and begin the planning process by clicking on the blue *Open* button next to the new plan

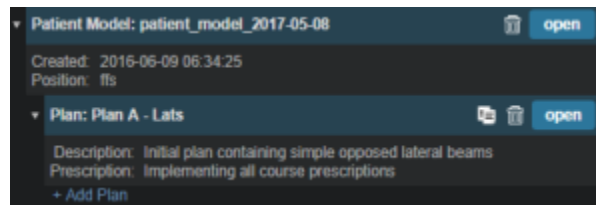

1. Note that users are free to have as many plans as desired within a the *Patient Model* and each *Plan* will specify which portion of the Course Prescriptions it is attempting to implement

From:

<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:creating_a_plan&rev=1497965769

Last update: **2021/07/29 18:24**

