


Creating a Plan

1. The point has now been reached where a *Plan* can be created
2. Click on the blue *Add Plan* link under the *Patient Model* to create a new plan
 1. In the box that opens the user should name the plan and add any description they may want

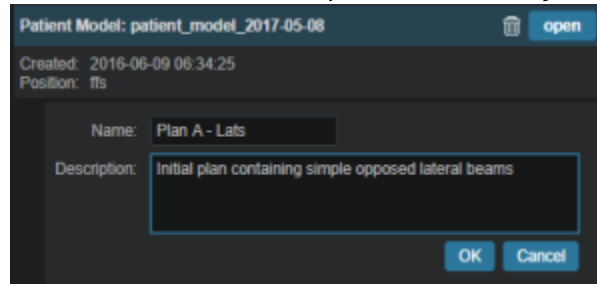


2. Note that the *Base Plan* option is used to specify whether an empty plan should be created or if the new plan should be pre-filled using the selected *Plan Template* (details on *Plan Templates* can be found [here](#))
 3. Click on the blue *OK* button when finished and the *Plan* has been created
3. Open the plan and begin the planning process by clicking on the blue *Open* button next to the new plan
 1. Note that users are free to have as many plans as desired within a the *Patient Model* and each *Plan* will specify which portion of the *Course Prescriptions* it is attempting to implement

Creating a Plan using a Plan Based on a Plan Template

Templates can be saved at either the site or user level. **All***italic Text* users will be able to access site level templates. Only the user who created them will be able to user level templates. Templates must be validated prior to the creating a new treatment plan to ensure compatibility with the new patient model

1. Click on the blue *Add Plan* link under the *Patient Model* to create a new plan
2. Choose the plan that should be used from the *Base Plan* drop down menu and click blue *OK*

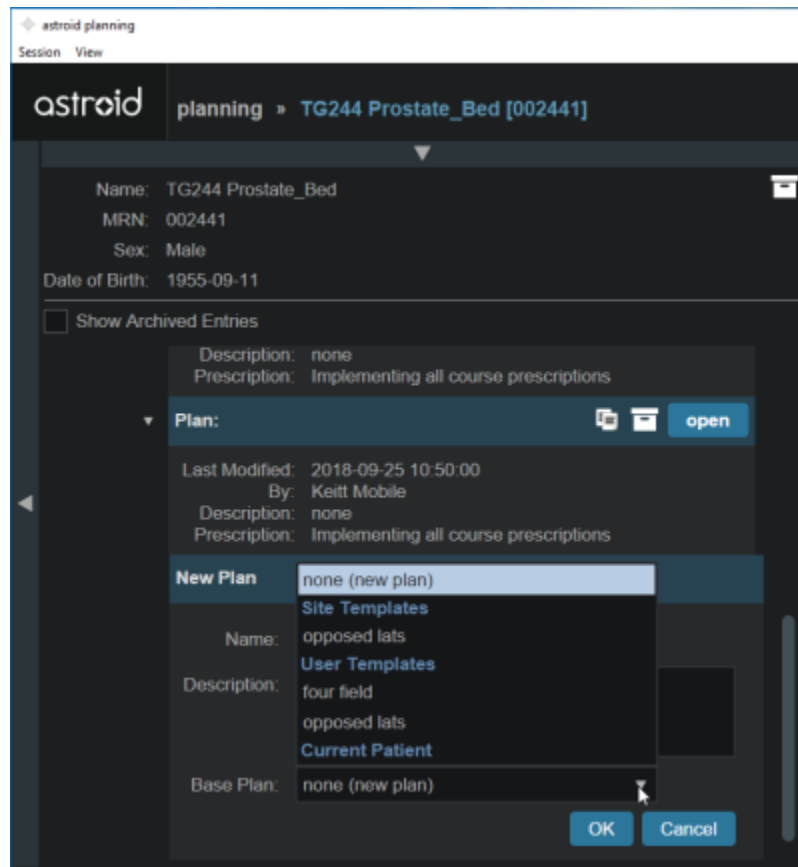

3. Make any changes necessary within plan and proceed with planning

From:
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:creating_a_plan&rev=1537977132

Last update: **2021/07/29 18:24**

