


Creating a Plan

1. The point has now been reached where a *Plan* can be created
2. Click on the blue *Add Plan* link under the *Patient Model* to create a new plan
 1. In the box that opens the user should name the plan and add any description they may want

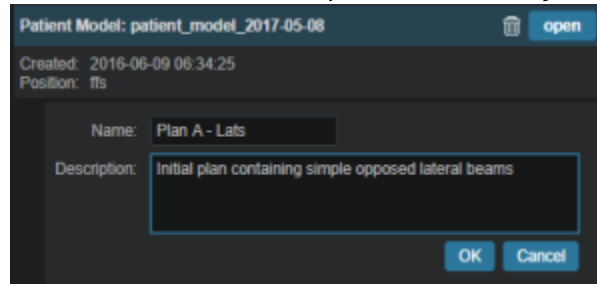

2. Note that the *Base Plan* option is used to specify whether an empty plan should be created or if the new plan should be pre-filled using the selected *Plan Template* (details on *Plan Templates* can be found [here](#))
 3. Click on the blue *OK* button when finished and the *Plan* has been created
3. Open the plan and begin the planning process by clicking on the blue *Open* button next to the new plan
 1. Note that users are free to have as many plans as desired within a the *Patient Model* and each *Plan* will specify which portion of the *Course Prescriptions* it is attempting to implement

Creating a Plan using a Plan Based on a Plan Template

Templates can be saved at either the site or user level. **All** *italic Text* users will be able to access site level templates. Only the user who created them will be able to user level templates. Templates must be validated prior to the creating a new treatment plan to ensure compatibility with the new patient model

1. Click on the blue *Add Plan* link under the *Patient Model* to create a new plan
2. Choose the plan that should be used from the *Base Plan* drop down menu and click blue *OK*

astroid planning
Session View

astroid planning > TG244 Prostate_Bed [002441]

Name: TG244 Prostate_Bed
MRN: 002441
Sex: Male
Date of Birth: 1955-09-11

Show Archived Entries

Description: none
Prescription: Implementing all course prescriptions

Plan:

Last Modified: 2018-09-25 10:50:00
By: Keitt Mobile
Description: none
Prescription: Implementing all course prescriptions

New Plan

- none (new plan)
- Site Templates
- User Templates
- four field
- opposed lats
- Current Patient

Name: opposed lats
Description: four field

Base Plan: none (new plan)

3. Make any changes necessary within plan and proceed with planning

From:
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:creating_a_plan&rev=1537977132

Last update: **2021/07/29 18:24**

