


Creating a Plan

1. The point has now been reached where a *Plan* can be created
2. Click on the blue *Add Plan* link under the *Patient Model* to create a new plan
 1. In the box that opens the user should name the plan and add any description they may want

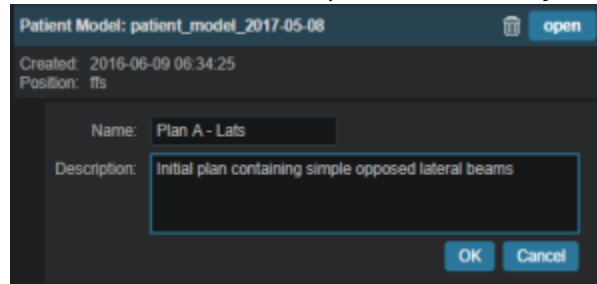

The screenshot shows a dialog box titled "Patient Model: patient_model_2017-05-08" with an "open" button. Below the title, it displays "Created: 2016-06-09 06:34:25" and "Position: ffs". The main area contains a "Name:" field with the text "Plan A - Lats" and a "Description:" field with the text "Initial plan containing simple opposed lateral beams". At the bottom right, there are "OK" and "Cancel" buttons.



2. Note that the *Base Plan* option is used to specify whether an empty plan should be created or if the new plan should be pre-filled using the selected *Plan Template* (details on *Plan Templates* can be found [here](#))
3. Click on the blue *OK* button when finished and the *Plan* has been created
3. Open the plan and begin the planning process by clicking on the blue *Open* button next to the new plan
 1. Note that users are free to have as many plans as desired within a the *Patient Model* and each *Plan* will specify which portion of the Course Prescriptions it is attempting to implement

From:
<https://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
https://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:creating_a_plan&rev=1537978811

Last update: **2021/07/29 18:24**