


Creating a Plan

1. The point has now been reached where a *Plan* can be created
2. Click on the blue *Add Plan* link under the *Patient Model* to create a new plan
 1. In the box that opens the user should name the plan and add any description they may want

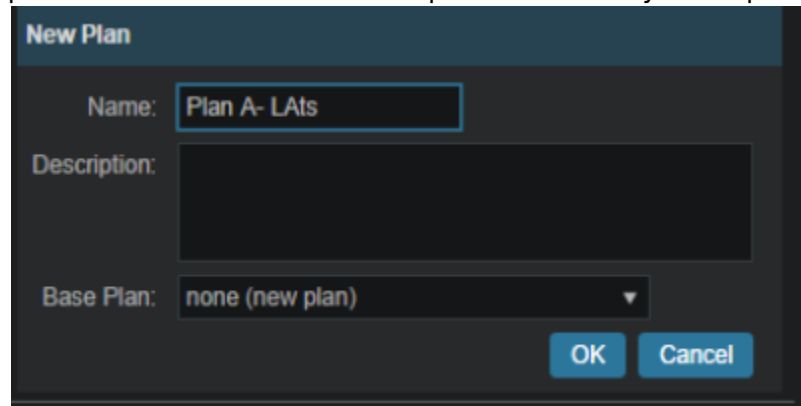

The screenshot shows a 'New Plan' dialog box with a dark background. It has three main input areas: 'Name' with the text 'Plan A- LATs', 'Description' which is an empty text area, and 'Base Plan' which is a dropdown menu currently showing 'none (new plan)'. At the bottom right, there are two buttons: 'OK' and 'Cancel'.

2. Note that the *Base Plan* option is used to specify whether an empty plan should be created or if the new plan should be pre-filled using the selected *Plan Template* (details on *Plan Templates* can be found [here](#))
3. Click on the blue *OK* button when finished and the *Plan* has been created
3. Open the plan and begin the planning process by clicking on the blue *Open* button next to the new plan
 1. Note that users are free to have as many plans as desired within a the *Patient Model* and each Plan will specify which portion of the Course Prescriptions it is attempting to implement

From:
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:creating_a_plan&rev=1537986012

Last update: **2021/07/29 18:24**

