Importing Patient Data

Now that a patient has been uploaded from DICOM to the Thinknode RKS, the Planning App should recognize that new patient files are available to import into a Planning patient.

1/3

- 1. Open the Astroid Launcher and launch the Planning App from your realm
- 2. Once Astroid Planning starts, click on the Imports Block in the task control pane on the left side

•						
► Patient Search						
Search for patients in the system.						
▼ Imports						
Search						
MRN:						
Name:						
Gender:	Any	×				
Show Archived Files						

- 3. Select the CT image set from the list of available files for import
- 4. Ensure that the MRN is correct
- 5. Click the Create Patient button to start the import process



- 6. Ensure that the date and time displayed in Astroid matched the current date and time in the current Windows OS.
- 7. Fill in the requested Patient Intent, taking care to select the appropriate *Treatment Site* as this selection contains the template information that will be used during structure set import
- 8. Select the appropriate HU to RSP curve (as shown below)

Costroid planning = Create	Section planning & Create Patient Information SERT PROSTATE 11111 DICCM Information View and archive DICCM information. Patient Data Teatment Data Patient Data Information Patient Data Information Server1, 120 Myr, FOW[6, 1000] Stacture Set Archive DICCM an import Cancel						
	Patient Information SBRT PROSTATE 11111 DICOM Information View and anchive DICOM information. Patient Data Inferst Data Inferst Data Instant Size* Protocol Protocol Narrative: Protocol Scanner2, 140 AVp; FOV [0, 1000] Stacture Set View AND DICOM on Import Archive DICOM on Import Inport Structure Scanner2, 140 AVp; FOV [0, 1000] Stacture Set Archive DICOM on Import Inport Structure Scanner2, 140 AVp; FOV [0, 1000] Stacture Set Cancel	astroid _{plan}	ning + Create				
	Patient Information SBRT PROSTATE 111111 DOCOM Information View and archive DICOM information. Patient Data Teatment Site *. Norrative: Protocol New Protocol RSP Data HU to RSP Curve*. Scanner1, 120 NUp, FOV [8, 1000] Structure Set User_curve-us Archive DICOM on import Import Structure						
DICCOM Information View and archive DICOM information. Patient Data Infent Data Treatment Site *: Narrative: Scarrer1, 120 kWp; FOV [0, 1000] Scarrer2, 140 kWp; FOV [0, 1000]	DICCOM Information View and archive DICCOM information. Patient Data Intern Data Treatment Site *. Narrative Protocol Protocol Protocol Protocol Sammer1, 120 AU(s; FOV [8, 1000] Stanture Set User 2019-00-05 View DICCOM on Import	Patient Information SBRT PROSTATE 111111					
View and archive DICOM information.	View and archive DICOM information.	DICOM Inform	ation				
Value and an other Cocces Produced Infrest Data Treatment Site *: Narrative: Protocol Protocol RSP Data HJ to RSP Curvet: Scanner1, 120 AV(0, FOV [0, 1000] Transition Scanner2, 140 AV(0, FOV [0, 1000] Transition	Variant Dota Infect Data Treatment Site* Narrative Protocol Protocol	blow and authin	a CACCAM information				
Patient Data Infrast Data Treatment Site*: Narrative: Protocol RSP Data HU to RSP Curve*: Scanner1: 120 M/g; FOV [0, 1000] Import Structure Scanner2: 140 M/g; FOV [0, 1000] Import Structure	Patient Data Inferst Data Teatment Site *: Narrative Protocol Protocol RSP Data HU to RSP Curret*: Scanner1: 120 kVp; FOV [0, 1000] Structure Set Scanner2: 140 kVp; FOV [0, 1000] * Archiva DICOM on import Import Cancel		LICON HUMANI				
Indext Data Treatment Site *: Norrative Protocol Protocol RSP Data HU to RSP Curve*: Scanner1: 120 AV(0; FOV [0, 1000] Timport Structure Scanner2: 140 AV(0; FOV [0, 1000]	Indext Data Treatment Site *: Narrative: Protocol Protocol RSP Data HU to RSP Curve*: Scanner1: 120 kV(p; FOV [0, 1000] Structure Scanner2: 140 kV(p; FOV [0, 1000] Structure Set Unter: 2010-00-09 * Archive DICOM on import	▼ Patient Deta					
Treatment Site *: Norrative: Protocol Protocol RSP Data HU to RSP Curve*: Scanner1: 120 AV(s; FOV [0, 1000] Tagoot Structure Scanner2: 140 AV(s; FOV [0, 1000]	Treatment Site *: Narrative Protocol FSP Duta HU to RSP Curve*: Scanner1: 120 kV(p; FOV [0, 1000] Structure Set Use: 2010-00-09 * Archive DICOM on import	Intent Data					
Narrative: Protocol RSP Data HU to RSP Convert: Import Structure Scanner1: 120 kV(p; FOV [0, 1000] Scanner2: 140 kV(p; FOV [0, 1000]	Narrative: Protocol RSP Data HU to RSP Convert. Import Structure Stanner2, 140 kVp; FOV (0, 1900) Scanner2, 140 kVp; FOV (0, 1900) Scanner2, 140 kVp; FOV (0, 1900) W Archive DICOM on import Import. Concol	Treatment Site *:					
Piolocol Piolocol RSP Data HU to RSP Curve*: Scanner1: 120 kVp; FOV [0, 1000] Scanner2: 140 kVp; FOV [0, 1000]	Pistocal Pistocal RSP Duta HU to RSP Curvet: Scanner1, 120 kVp; FOV [0, 1000] Structure Set Scanner2, 140 kVp; FOV [0, 1000] User: 2010-00-09 Archiva DICOM on import	Alexander					
RSP Data HU to RSP Curvet: Import Structure Scanner1: 120 kVp; FOV [0, 1000] Scanner2: 140 kVp; FOV [0, 1000]	RSP Data HU to RSP Curvet: Import Structure Scanner1: 120 kVp; FOV [0, 1000] Stucture Set Unter: 2010-00-09 Active DICOM on import	Parkent	New Protocol				
RSP Data HU to RSP Curvet: Scanner1: 120 kVp; FOV [0, 1000] Scanner2: 140 kVp; FOV [0, 1000]	RSP Data HU to RSP Curvet: Import Structure Scanner1: 120 kVp; FOV [0, 1000] Shuchare Set Unit: 2/19-09-09 * Archive DICOM on import Import Concol	1 1000000					
HU to RSP Curve*: Import Structure Scanner1: 120 kVp; FOV [0, 1000] Scanner2: 140 kVp; FOV [0, 1000]	HU to RSP Curve*: Import Structure Scanner1, 120 kVp; FOV [0, 1000] Structure Set Unit: 2019-09-09 Archive DICOM on import Import Cancel	RSP Data					
Import Structure Scanner2: 140 kVp; FOV [0, 1000]	Import Structure Scanner2, 140 kVp; FOV [0, 1000] Structure Set Archiva DICOM on import Import Cancel	HU to RSP Curver:					
Scanner2; 140 kVp; FOV [0, 1000]	Scanner2, 140 kVp; FOV [0, 1000] Shuchare Set Use: 2019-09-09 V Archive DICOM on import Import Cancel	Import Structure	Scanner1; 120 kVp; FOV [0, 1000]				
	Archive DICOM on import	Import out to the	Scanner2; 140 kVp; FOV [0, 1000]				
Structure Set Little 2010-00-00	Archive DICOM on import	Structure Set	UBIK 2010-00-09 *				

- The corresponding structure set (SS) file to import with these images will automatically be selected. The structures will show up below the Patient Data box in the Import structures box (note that the available choices will be automatically filtered based on the structure set DICOM UID information)
 - 1. The structures associate with the data set be available as a list of the available structures
 - 2. Here you may choose whether or not to import each structure by checking or unchecking the box beside each structure name
 - 3. Matched, Assigned, and Custom structures are designated with corresponding tags at the end of the structure name in the structure list
 - 1. You may only edit structures that are shown as *Custom*, which indicates the name did not exactly match a directive structure from the *Treatment Site* template selected above
 - 2. For all custom structures, the type is by default set to "Other", unless it contains the letters "TV" (as in PTV or CTV), in which case it is assigned the type of "Target"; the type may be changed here if needed
 - 3. Alternatively you may *Assign* a *Custom* structure to a directive level template structure using the provided drop down menu (this is useful when structure names contain typos or contour names otherwise do not match your standard site protocols)
 - 1. Assigning a custom structure to a defined directive structure will result in the imported structure inheriting all the predefined structure properties (e.g. name, type, color)

3/3

× Bowel (custom)			
Assign Site Structure:	<custom structure=""></custom>	•	
New Directive Structure:			
Type:	Other •		
Description:			

- 10. Once all structures have been selected, assigned, and edited as needed, click the *Import* button to create the patient and import the CT Images and Structures into it
- 11. The patient is now created and all available data has been imported
- 12. Click on the Back to Import button to return back to the Imports task

From: http://apps.dotdecimal.com/ - decimal App Documentation Permanent link: http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:importing&rev=1501775963 Last update: 2021/07/29 18:24