
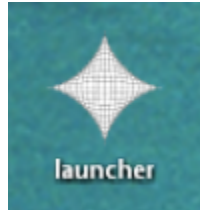


Launching Astroid

1. Open the Astroid Launcher

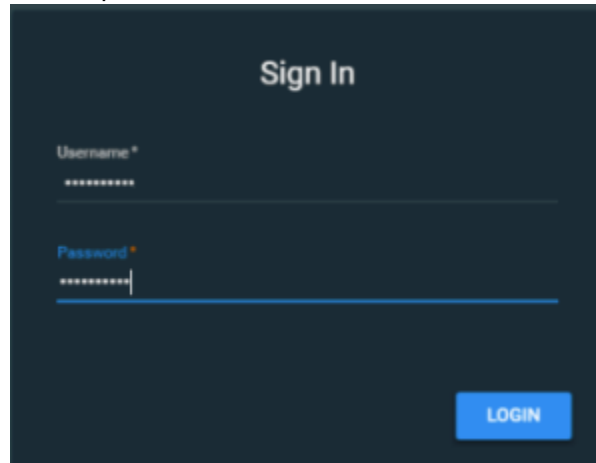


2. Sign in using your user name and password

The image shows a "Sign In" screen with a dark blue background. At the top, the text "Sign In" is displayed in white. Below it, there are two input fields: "Username*" and "Password*", both with white text and a small asterisk. The "Username*" field has a white underline and a white cursor. The "Password*" field has a white underline and a white cursor. Below the "Password*" field, there is a blue button with the text "LOGIN" in white.


3. Select the blue *Log In* button
4. Select your realm from the list of available realms

The image shows a "Realm Selection" screen with a dark blue background. At the top, the text "Realm Selection" is displayed in white. Below it, there is a list of realms, each with a name and a description. The realms are: "dev-anegron" (The development realm for Alfredo Negron-Rivera at .decimal.), "dev-aclark" (The development realm for Adam Clark at .decimal.), "dev-kmobile" (The development realm for Keitt Mobile at .decimal.), and "dev-staging" (The realm for the staging branch of work.). A red arrow points to the "dev-kmobile" realm.

5. From the left side tool bar select the application you would like to launch

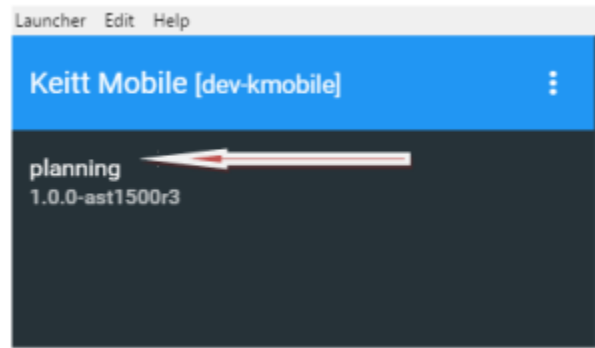

6. Select the blue *Launch* button

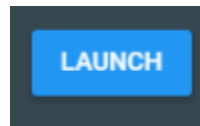

7. If there is an updated version of this application the *Launch* button will say *Install*. Click this and allow the latest version to install. After the latest version is installed the blue button will say *Launch*
8. You may now proceed with opening a current patient or importing a new patient

From:

<https://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

<https://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:launcher&rev=1484167753>

Last update: **2021/07/29 18:24**