



Launching Astroid

The Astroid Launcher will house various applications that may be used as part of the Astroid Treatment Planning system. Any updates to these applications will automatically be deployed to the launcher. The user will be notified that there is an update or a new version that will need installation once they have chosen an application.


1. Open the Astroid Launcher

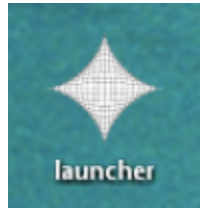


2. Sign in using your user name and password

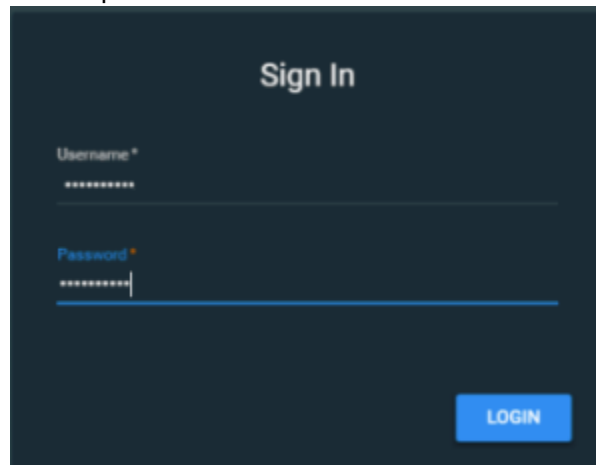The image shows a "Sign In" form with a dark blue background. At the top, the text "Sign In" is displayed in white. Below it, there are two input fields: "Username*" and "Password*", both with white text and a blue outline. The "Password*" field has a blue eye icon to its right. A blue "LOGIN" button is located at the bottom right of the form.


3. Select the blue *Log In* button
4. Select your realm from the list of available realms

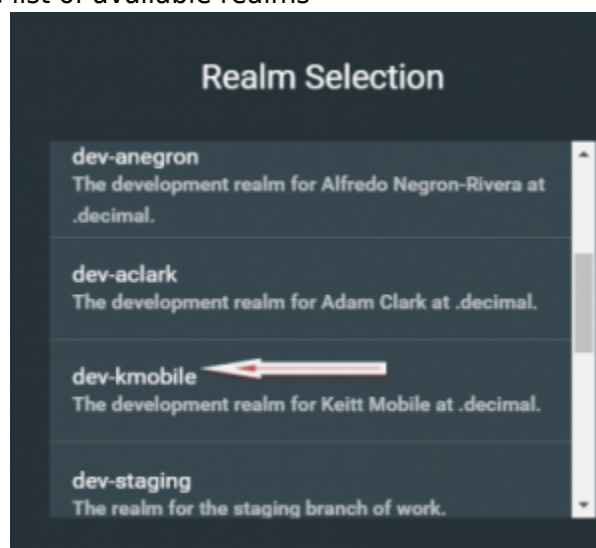The image shows a "Realm Selection" form with a dark blue background. At the top, the text "Realm Selection" is displayed in white. Below it, there is a list of realms, each with a name and a description. The realms are: "dev-anegron" (The development realm for Alfredo Negron-Rivera at .decimal.), "dev-aclark" (The development realm for Adam Clark at .decimal.), "dev-kmobile" (The development realm for Keitt Mobile at .decimal.), and "dev-staging" (The realm for the staging branch of work.). A red arrow points to the "dev-kmobile" realm.

5. From the left side tool bar select the application you would like to launch

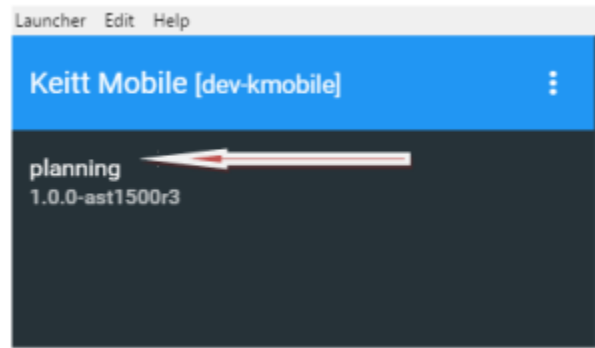

6. Select the blue *Launch* button

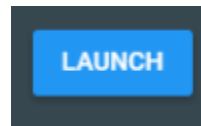

7. If there is an updated version of this application the *Launch* button will say *Install*. Click this and allow the latest version to install. After the latest version is installed the blue button will say *Launch*
8. You may now proceed with opening a current patient or importing a new patient

From:

<http://apps.dotdecimal.com/> - decimal App Documentation

Permanent link:

<http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:launcher&rev=1484169766>

Last update: 2021/07/29 18:24

