



# Launching Astroid

The Astroid Launcher will house various applications that may be used as part of the Astroid Treatment Planning system. Any updates to these applications will automatically be deployed to the launcher. The user will be notified that there is an update or a new version that will need installation once they have chosen an application.

1. Open the Astroid Launcher

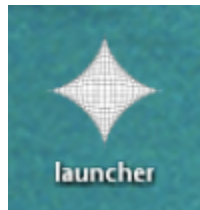


2. Sign in using your user name and password

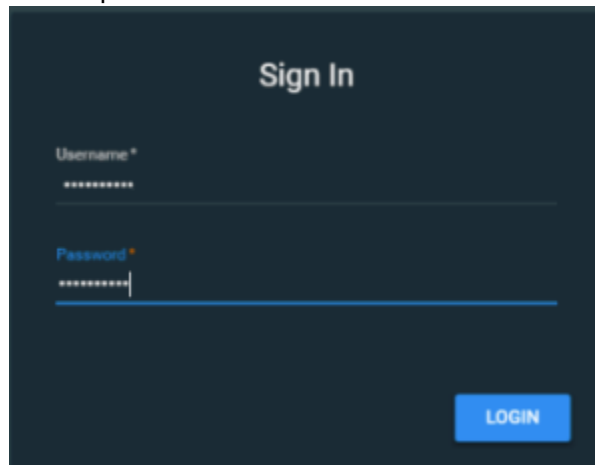


3. Select the blue *Log In* button
4. Select your realm from the list of available realms



5. From the left side tool bar select the application you would like to launch

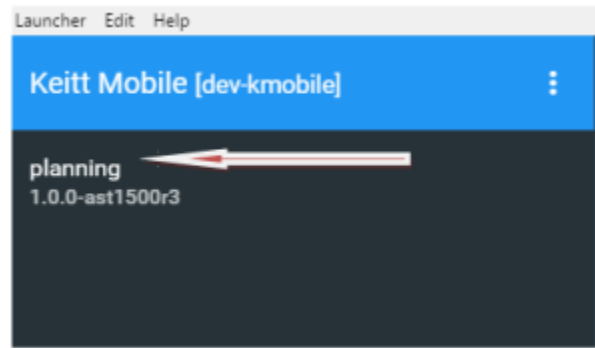

6. Select the blue *Launch* button

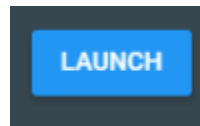

7. If there is an updated version of this application the *Launch* button will say *Install*. Click this and allow the latest version to install. After the latest version is installed the blue button will say *Launch*
8. You may now proceed with opening a current patient or importing a new patient

From:  
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:  
<http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:launcher&rev=1484169766>

Last update: **2021/07/29 18:24**

