2025/11/06 09:57 1/2 Launching Astroid


## **Launching Astroid**

The Astroid *Launcher* will house the various applications that will be used as part of the Astroid Treatment Planning system. Any updates to these applications will automatically be deployed to the *Launcher*. The user will be notified that there is an update or a new version that will need installation once they have chosen an application.


1. Open the Astroid Launcher



2. Sign in using your user name and password

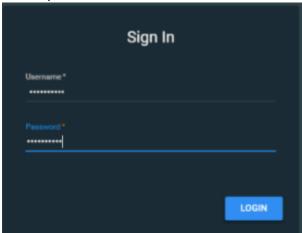


- 3. Select the blue *Log In* button
- 4. Select your realm from the list of available realms

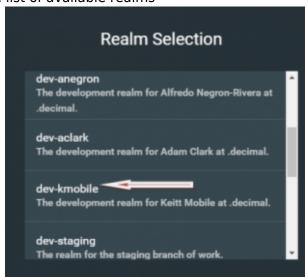

5. From the left side tool bar select the application you would like to launch

2025/11/06 09:57 2/2 Launching Astroid

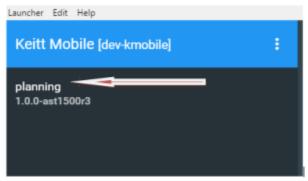

6. Select the blue Launch button



- 7. If there is an updated version of this application the *Launch* button will say *Install*. Click this and allow the latest version to install. After the latest version is installed the blue button will say *Launch*
- 8. You may now proceed with opening a current patient or importing a new patient

## From:

http://apps.dotdecimal.com/ - decimal App Documentation

Permanent link:

http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:launcher&rev=1484169835

Last update: 2021/07/29 18:24

