



# Launching Astroid

The *Astroid Launcher* will house the various applications that will be used as part of the Astroid Treatment Planning system. Any updates to these applications will automatically be deployed to the *Launcher*. The user will be notified that there is an update or a new version that will need installation once they have chosen an application.


1. Open the *Launcher*

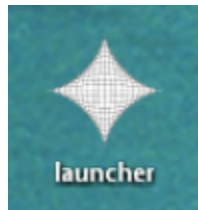


2. Sign in using your user name and password

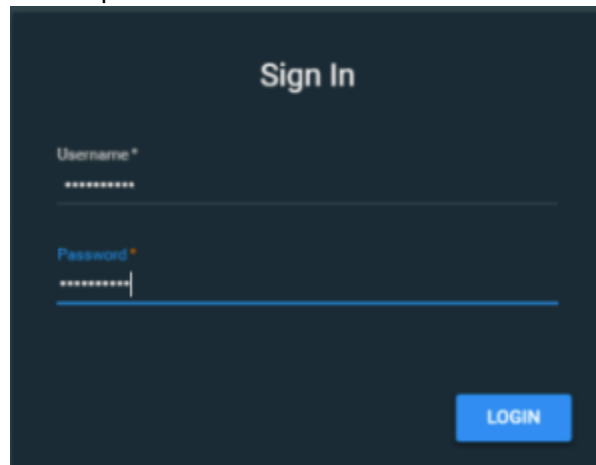


3. Select the blue *Log In* button
4. Select your realm from the list of available realms

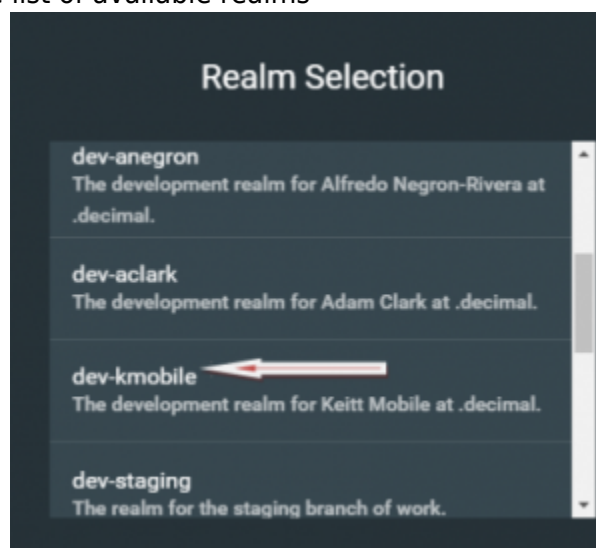

5. From the left side tool bar select the application you would like to launch

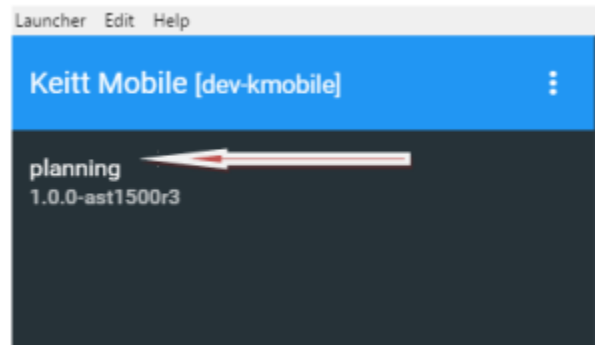

6. Select the blue *Launch* button

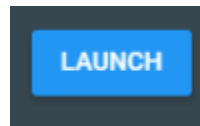

7. If there is an updated version of this application the *Launch* button will say *Install*. Click this and allow the latest version to install. After the latest version is installed the blue button will say *Launch*
8. You may now proceed with opening a current patient or importing a new patient

From:  
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:  
<http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:launcher&rev=1484169877>

Last update: **2021/07/29 18:24**

