2025/11/06 10:18 1/2 Launching Astroid


Launching Astroid

The Astroid *Launcher* will house the various applications that will be used as part of the Astroid Treatment Planning system. Any updates to these applications will automatically be deployed to the *Launcher*. The user will be notified that there is an update or a new version that will need installation once they have chosen an application.

1. Open the Launcher



2. Sign in using your user name and password

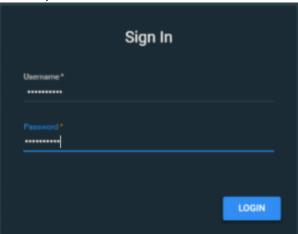




new pic w/account

- 3. Click the blue Log In button
- 4. Select your realm from the list of available realms



5. From the left side tool bar select the application you would like to launch

2025/11/06 10:18 2/2 Launching Astroid

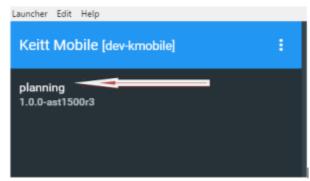

6. Click the blue Launch button



- 1. If there is an updated version of this application, the *Launch* button will not appear and instead an *Install* button will be available. Click this to allow the latest version to install. After the latest version is installed, the blue button will revert back to *Launch*, which you can now click
- 7. This will open the Astroid Planning application and bring you directly to the main patient search screen
- 8. You may now proceed with opening a current patient or importing a new patient

From:

http://apps.dotdecimal.com/ - decimal App Documentation

Permanent link:

http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:launcher&rev=1497898847

Last update: 2021/07/29 18:24

