



Launching Astroid

The Astroid *Launcher* will house the various applications that will be used as part of the Astroid Treatment Planning system. Any updates to these applications will automatically be deployed to the *Launcher*. The user will be notified that there is an update or a new version that will need installation once they have chosen an application. Use the following steps to open the *Launcher* and launch the Astroid Planning App:

1. Open the *Launcher*

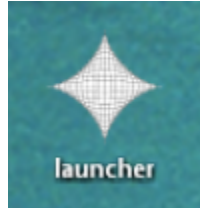

2. Sign in using your user name and password

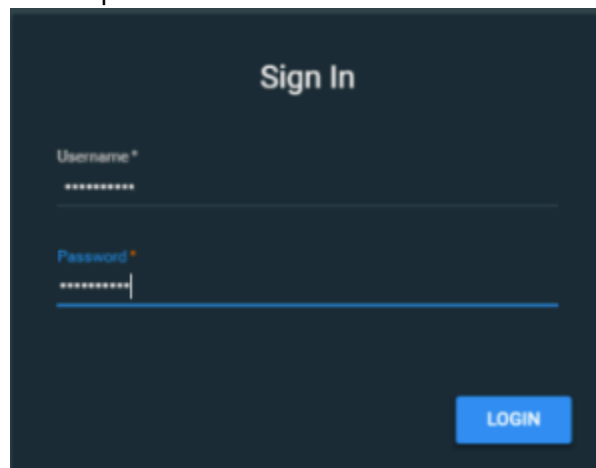
The image shows a "Sign In" form with a dark blue background. It has two input fields: "Username*" and "Password*", both with masked text (dots). A blue "LOGIN" button is located at the bottom right.

new pic w/account


3. Click the blue *Log In* button
4. Select your realm from the list of available realms

The image shows a "Realm Selection" screen with a dark blue background. It lists four realms: "dev-anegron", "dev-aclark", "dev-kmobile", and "dev-staging". Each realm has a description. A red arrow points to the "dev-kmobile" realm.

5. From the left side tool bar select the application you would like to launch

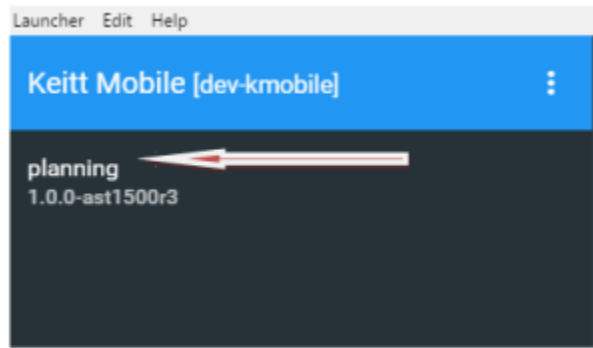

6. Click the blue *Launch* button

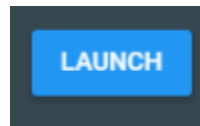

1. If there is an updated version of this application, the *Launch* button will not appear and instead an *Install* button will be available. Click this to allow the latest version to install. After the latest version is installed, the blue button will revert back to *Launch*, which you can now click
7. This will open the Astroid Planning application and bring you directly to the main patient search screen
8. You may now proceed with opening a current patient or importing a new patient

From:

<http://apps.dotdecimal.com/> - decimal App Documentation

Permanent link:

<http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:launcher&rev=1498048228>

Last update: **2021/07/29 18:24**

