


# Launching Astroid

The Astroid *Launcher* will house the various applications that will be used as part of the Astroid Treatment Planning system. Any updates to these applications will automatically be deployed to the *Launcher*. The user will be notified that there is an update or a new version that will need installation once they have chosen an application. Use the following steps to open the *Launcher* and launch the Astroid Planning App:

1. Open the *Launcher*

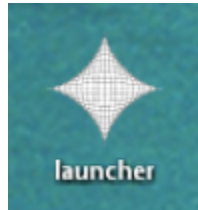

2. Sign in using your user name and password


The image shows a dark blue "Sign In" screen. It has two input fields: "Username\*" and "Password\*", both with asterisks indicating they are required. Below the "Password\*" field is a blue "LOGIN" button.

new pic w/account


3. Click the blue *Login* button
4. Select your realm from the list of available realms

The image shows a "Realm Selection" screen with a list of development realms. The realms listed are: "dev-anegron" (The development realm for Alfredo Negron-Rivera at .decimal.), "dev-aclark" (The development realm for Adam Clark at .decimal.), "dev-kmobile" (The development realm for Keitt Mobile at .decimal.), and "dev-staging" (The realm for the staging branch of work.). A white arrow points to the "dev-kmobile" option.

5. From the left side tool bar select the application you would like to launch

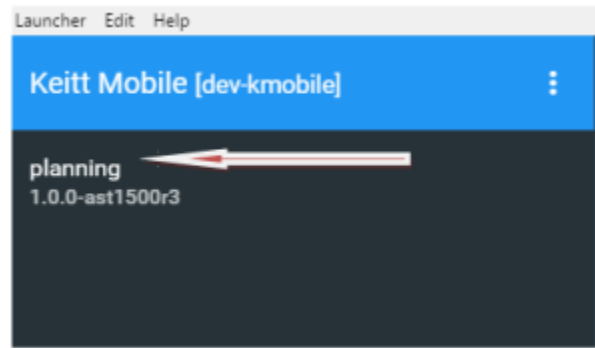

6. Click the blue *Launch* button

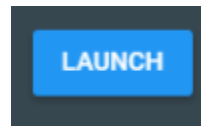

1. If there is an updated version of this application, the *Launch* button will not appear and instead an *Install* button will be available. Click this to allow the latest version to install. After the latest version is installed, the blue button will revert back to *Launch*, which you can now click
7. This will open the Astroid Planning application and bring you directly to the main patient search screen
8. You may now proceed with opening a current patient or importing a new patient

From:  
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:  
<http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:launcher&rev=1498048278>

Last update: **2021/07/29 18:24**

