



# Launching Astroid

The Astroid *Launcher* will house the various applications that will be used as part of the Astroid Treatment Planning system. Any updates to these applications will automatically be deployed to the *Launcher*. The user will be notified that there is an update or a new version that will need installation once they have chosen an application. This will ensure that cloud version of Launcher and the local version of Launcher remain synchronized. Use the following steps to open the *Launcher* and launch the Astroid Planning App:


1. Open the *Launcher*

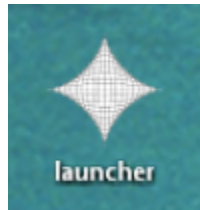


2. Sign in using your account, user name and password

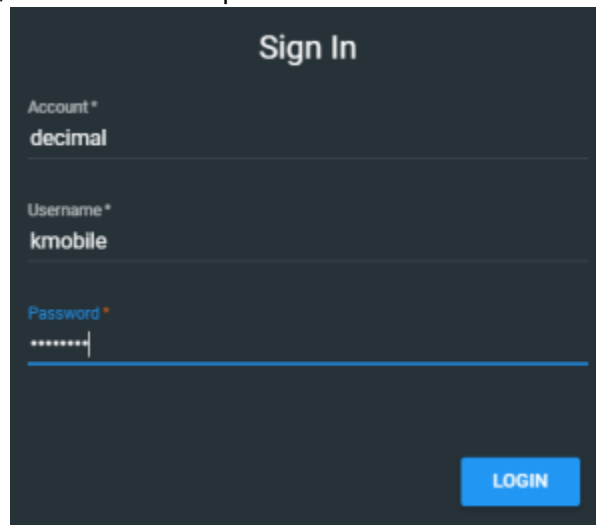The image shows a "Sign In" screen with a dark background. At the top, the text "Sign In" is centered in white. Below it, there are three input fields: "Account \*" with the value "decimal", "Username \*" with the value "kmobile", and "Password \*" with a masked password ".....". A blue "LOGIN" button is located at the bottom right of the form.


3. Click the blue *Login* button
4. Select your realm from the list of available realms

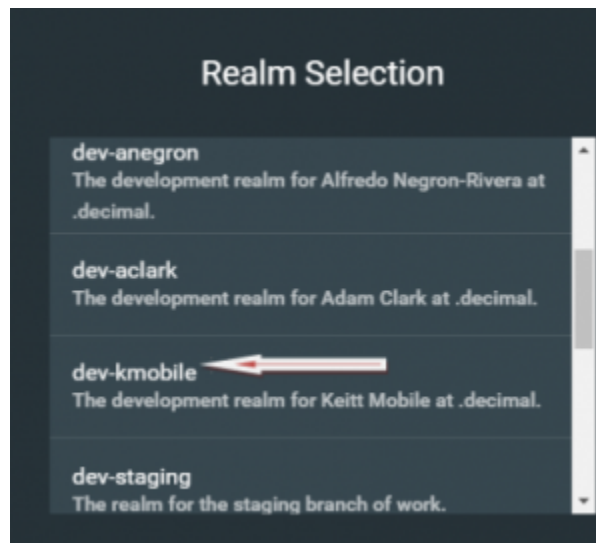The image shows a "Realm Selection" screen with a dark background. At the top, the text "Realm Selection" is centered in white. Below it, there is a list of realms. The first three are "dev-anegron", "dev-aclark", and "dev-kmobile". The "dev-kmobile" realm is selected, indicated by a white arrow pointing to it from the left. Below "dev-kmobile" is "dev-staging". Each realm entry includes a description: "The development realm for Alfredo Negron-Rivera at .decimal.", "The development realm for Adam Clark at .decimal.", "The development realm for Keitt Mobile at .decimal.", and "The realm for the staging branch of work." respectively.

5. From the left side tool bar select the application you would like to launch

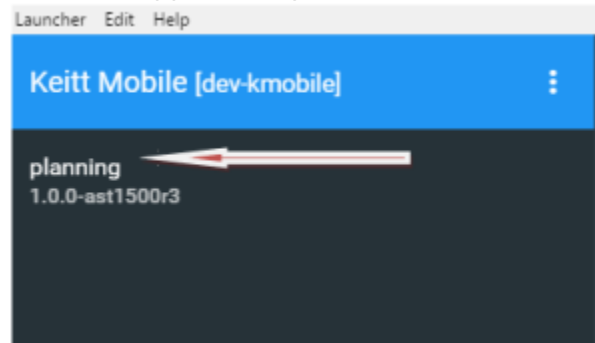

6. Click the blue *Launch* button

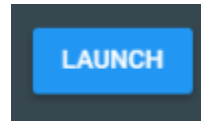

1. If there is an updated version of this application, the *Launch* button will not appear and instead an *Install* button will be available. Click this to allow the latest version to install. After the latest version is installed, the blue button will revert back to *Launch*, which you can now click

7. This will open the Astroid Planning application and bring you directly to the main patient search screen

8. You may now proceed with opening a current patient or importing a new patient

From:

<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

<http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:launcher&rev=1501606213>

Last update: **2021/07/29 18:24**

