1/3

Uploading a DICOM Patient

At this time Astroid only accepts CT data sets

1. Open the Launcher and choose the appropriate realm and application to work in



2. Choose the Imports block and Blue Browse button in the Upload Files section

astroid planning				
Session View				
astroid planning				
▼				
Patient Search				
Search for patients in the system.				
▼ Imports				
 Search Files 				
▼ Upload Files				
Select file(s) for upload				
	Browse			
Selected file(s):				
	Upload Stop			

3. Navigate to the directory where the DICOM images and structure set are stored

astraid planning Patient Search of Directory V los Search Files Artroid ad Files DICOM CERR Bowse MOH-patient MR-MatchFiles 16344 and Shop DCM_TG244_abdox DCM_TG244,enus DCM_TG244_h8in DCM_TG244_hang DCM_T0244_prostat UFHO MRT with Images MEPCC Anon (Head) DCM_35244_proct hiden OK Canor Make New Folder

 Choose the appropriate DICOM images and structure set and and choose the blue Upload button in the bottom right corner. The DICOM files will populate in the block. This may take a couple of minutes to complete

0	astroid	planning			
		▼			
	Patient Search				
	Search to	or patients in the system.			
	▼ Imports				
	Search	Files			
	▼ Upload	Files			
•	Select file(s)	for upload			
	C:\SVN ate	data\Astroid\DICOM\TG244\DCM_TG244_pros	t Browse		
	Selected file(s): 91		-	
	CT.2.16.8	840.1.113662.2.12.0.3173.1271873797.101.dcr	n		
	CT 2 16 8	940.1.113662.2.12.0.3173.1271673797.106.0cf 940.1.113662.2.12.0.3173.1271873797.111.dcr	n		
	CT.2.16.8	840.1.113662.2.12.0.3173.1271873797.116.dcr	n		
	CT.2.16.8	340.1.113662.2.12.0.3173.1271873797.121.dcr	n		
	CT.2.16.8	340.1.113662.2.12.0.3173.1271873797.126.dcr	n		
	CT 2 16.8	840 1 113662 2 12 0 3173 1271873797 136 der	" n		
	CT.2.16.8	840.1.113662.2.12.0.3173.1271873797.141.dcr	n		
	CT.2.16.8	340.1.113662.2.12.0.3173.1271873797.146.dcr	n		
	CT.2.16.8	840.1.113662.2.12.0.3173.1271873797.151.dcr	n		
	CT 2 16.0	\$40.1.113002.2.12.0.3173.1271873797.150.00 \$40.1.113662.2.12.0.3173.1271873797.161.des	n n		
	CT.2.16.8	840.1.113662.2.12.0.3173.1271873797.166.dcr	n		
	CT.2.16.8	840.1.113662.2.12.0.3173.1271873797.171.dcr	n		
	CT.2.16.8	340.1.113662.2.12.0.3173.1271873797.176.dcr	n		
	CT 2 16.0	840.1.113662.2.12.0.3173.1271873797.181.0cf 840.1.113662.2.12.0.3173.1271873797.186.dec	n		
	CT.2.16.8	840.1.113662.2.12.0.3173.1271873797.191.dcr	n		
	CT.2.16.8	340.1.113662.2.12.0.3173.1271873797.196.dcr	n		
	CT.2.16.8	340.1.113662.2.12.0.3173.1271873797.201.dcr	n		
			Upload	Stop	

5. Once the file has finished uploading the import process may start

From: http://apps.dotdecimal.com/ - decimal App Documentation

Permanent link:

http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:patient_import&rev=1537541180

3/3

Last update: 2021/07/29 18:25

