


# Uploading a DICOM Patient

\*At this time Astroid only accepts CT data sets\*


1. Open the Launcher and choose the appropriate realm and application to work in




2. Choose the Imports block and Blue Browse button in the Upload Files section

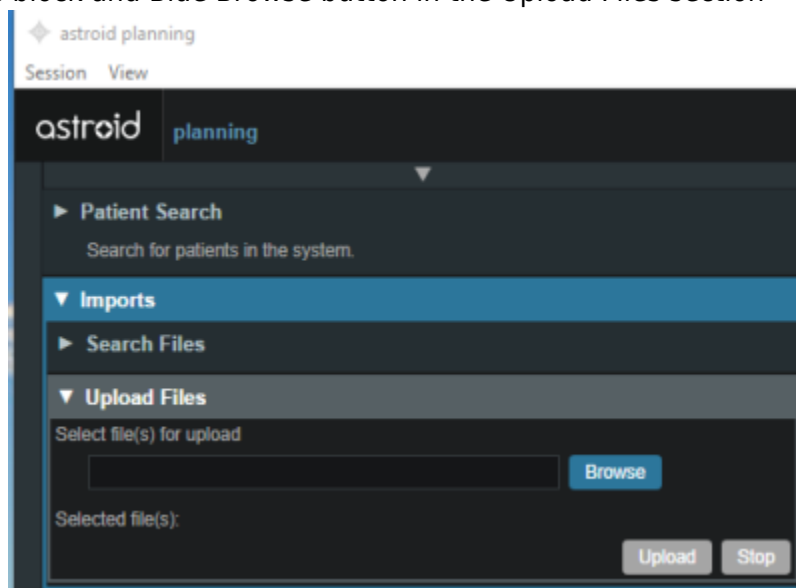

3. Navigate to the directory where the DICOM images and structure set are stored

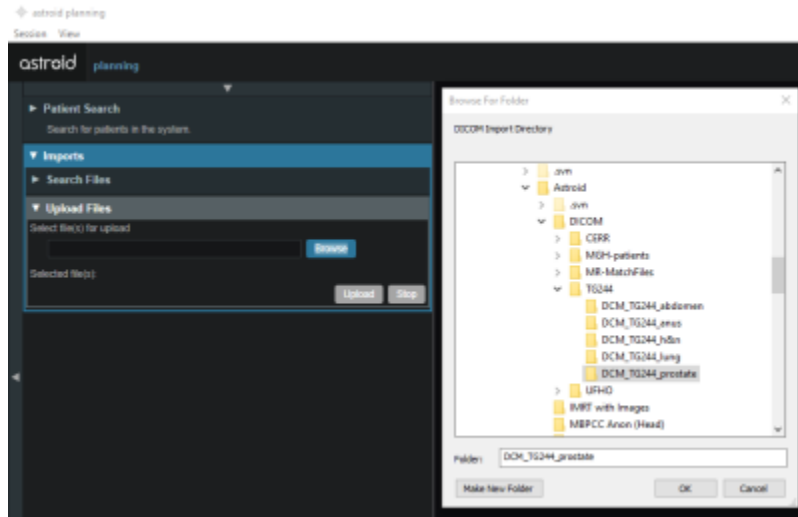

- 4. Choose the appropriate DICOM images and structure set and and choose the blue Upload button in the bottom right corner. The DICOM files will populate in the block. This may take a couple of minutes to complete

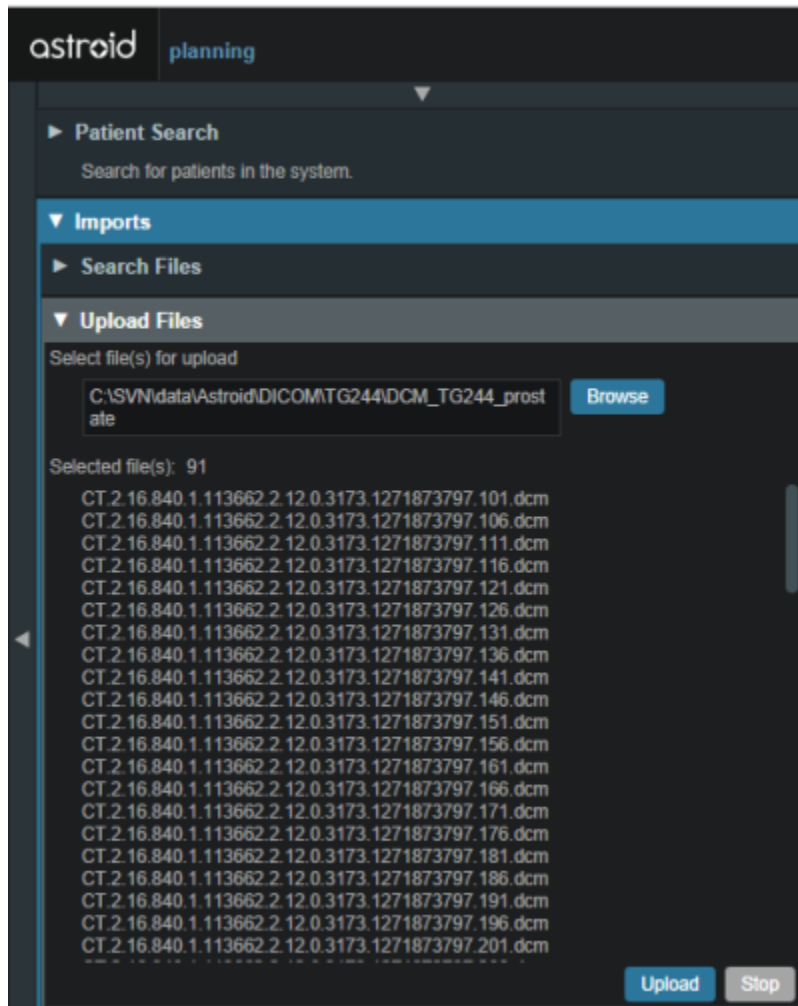

- 5. Once the file has finished uploading the import process may start

From:

<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

[http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:patient\\_import&rev=1537541180](http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:patient_import&rev=1537541180)

Last update: **2021/07/29 18:25**

