


Plan Export

Once a plan has been completed and the physician has approved the plan; the plan is ready to be sent to the Record and Verify system.

- Open the *Export* block.
- The qualified person box must be checked in order to export
- Choose which items to be exported
- Choose the location to export to- either locally or to a server

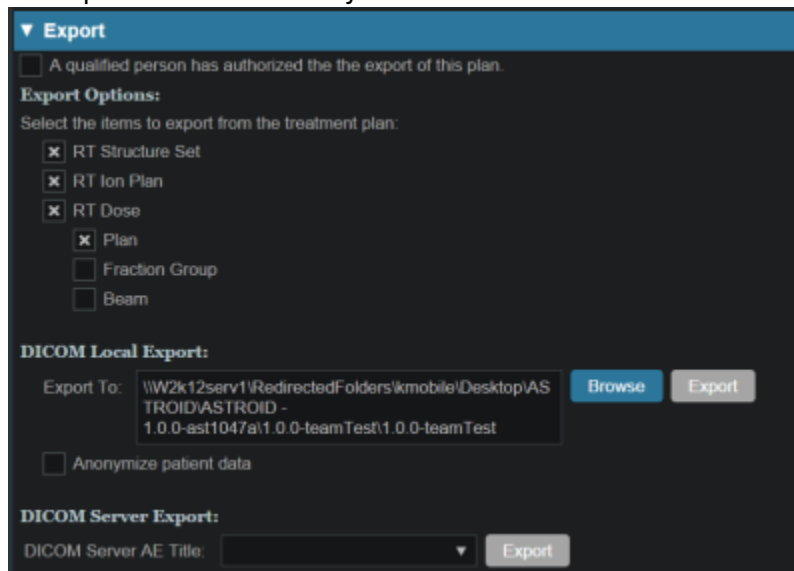

The screenshot shows the 'Export' section of a software interface. At the top, there is a blue header with a dropdown arrow and the word 'Export'. Below this, there is a checkbox labeled 'A qualified person has authorized the the export of this plan.' followed by the heading 'Export Options:'. Under 'Export Options', there is a sub-heading 'Select the items to export from the treatment plan:' and a list of items with checkboxes: 'RT Structure Set', 'RT Ion Plan', 'RT Dose', 'Plan', 'Fraction Group', and 'Beam'. The 'RT Structure Set', 'RT Ion Plan', 'RT Dose', and 'Plan' checkboxes are checked. Below this is the 'DICOM Local Export:' section, which includes an 'Export To:' field containing a file path, a 'Browse' button, and an 'Export' button. There is also an 'Anonymize patient data' checkbox. At the bottom is the 'DICOM Server Export:' section, which includes a 'DICOM Server AE Title:' dropdown menu and an 'Export' button.

From:
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:plan_export&rev=1538661746

Last update: **2021/07/29 18:25**

