


Plan Export

Once a plan has been completed and the physician has approved the plan; the plan is ready to be sent to the Record and Verify system.

- Open the *Export* block.
- The qualified person box must be checked in order to export
- Choose which items to be exported
- Choose the location to export to- either locally or to a server

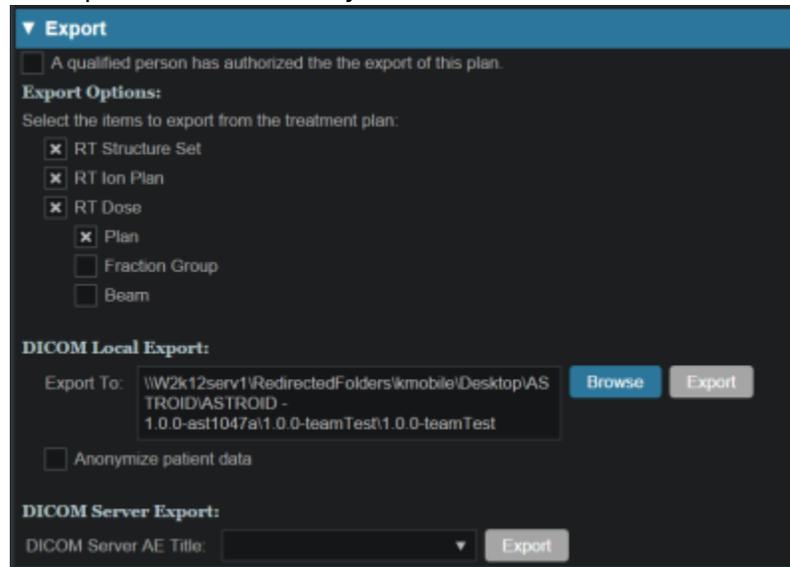

Export

A qualified person has authorized the the export of this plan.

Export Options:

Select the items to export from the treatment plan:

- RT Structure Set
- RT Ion Plan
- RT Dose
- Plan
- Fraction Group
- Beam

DICOM Local Export:

Export To:

Anonymize patient data

DICOM Server Export:

DICOM Server AE Title:

From:
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:plan_export&rev=1538661746

Last update: **2021/07/29 18:25**

