

Plan Export

Once a plan has been completed and the physician has approved the plan; the plan is ready to be sent to the Record and Verify system.

- Open the *Export* block.
- The qualified person box must be checked in order to export
- Choose which items to be exported
- Choose the location to export to- either locally or to a server
- When you choose to export locally the user has a choice to anonymize the data by checking the *Anonymize patient data* box under the *DICOM Local Export* option

Export

☐ A qualified person has authorized the the export of this plan.

Export Options:

Select the items to export from the treatment plan:

- ☒ RT Structure Set
- ☒ RT Ion Plan
- ☒ RT Dose
- ☒ Plan
- ☐ Fraction Group
- ☐ Beam

DICOM Local Export:

Export To: \\W2k12serv1\Redirection\Folders\kmobile\Desktop\ASTROID\ASTROID - 1.0.0-ast1047a\1.0.0-teamTest\1.0.0-teamTest **Browse** **Export**

☐ Anonymize patient data

DICOM Server Export:

DICOM Server AE Title: **Export**

From:
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:plan_export&rev=1538742386

Last update: **2021/07/29 18:25**

