
2025/11/03 18:11 1/1 Plan Export

Plan Export

Once a plan has been completed and the physician has approved the plan; the plan is ready to be sent to the Record and Verify system.

- Open the Export block.
- The qualified person box must be checked in order to export
- Choose which items to be exported
- Choose the location to export to- either locally or to a server
- When you choose to export locally the user has a choice to anonymize the data by checking the Anonymize patient data box under the DICOM Local Export option

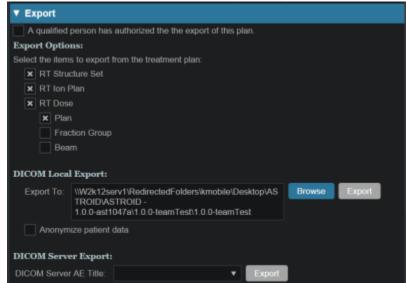

From:

http://apps.dotdecimal.com/ - decimal App Documentation

Permanent link:

http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:plan_export&rev=1538742386

Last update: 2021/07/29 18:25

